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EN Nasal Aspirator soft and easy and soft nozzles

Please read the instructions before use.

Validity: 5 years

Nasal hygiene is especially important in very young babies because in the first few months of life, a baby breathes mainly through its nose and doesn't know how to blow it yet. Babies cannot blow their noses and this affects their sleep, their feeding, and their own general comfort and that of their family. **If baby's nose is blocked, he will suffer from a real discomfort when eating, drinking or sleeping.** PhysioClean nasal aspirator allows the elimination of nasal secretions that can bother the baby. It offers an immediate solution for mothers to keep the nose unblocked and allow the baby to breathe freely, making it easier to sleep and feed. Its **ergonomic shape** with grip helps mother and **the nozzles**, made of soft and flexible materials, fit for baby's sensitive and delicate nose. The disposable nozzle, with pre-inserted absorbent filter, ensures **maximum hygiene**. It is a medical device. Store in a dry, cool place.

It contains:

- A. Nasal aspirator body with ergonomic handle and soft grip.
B. Bottom part of nasal aspirator.
C.3 Disposable anatomical nozzles with ANTI OCCLUSION fissures (Patented system), sealed individually.
D. Flexible tube.
E. Aspirating mouthpiece.
F. Handy carrying case.

INSTRUCTIONS FOR USE

- 1) Clean your hands thoroughly before use.
- 2) Before using for the first time, and after each use, wash the aspirator body and connector with warm water and a neutral soap, rinse well and leave out to dry.
- 3) Join the aspirating mouthpiece to the flexible tube.
- 4) Join the other end of the tube to the joining ring.
- 5) Insert the filter in the appropriate housing of the anatomical nozzle.
- 6) Join and fix the anatomical nozzle to the joining ring.
- 7) Squeeze some Physioclean Physiological Solution into the child's nose following the instructions provided.
- 8) Insert the tip of the nozzle into the baby's nostril.
- 9) Regularly aspirate from the mouthpiece until all the mucous has been removed. Repeat the same operation with the other nostril.
- 10) After each use remove the anatomical nozzle, with filter, and throw it away.

WARNINGS

Check that the detachable parts are fitted correctly. Only use in the presence of an adult and keep out of reach of children. The product contains small parts that can be swallowed. Warning! Suffocation risks. Store at room temperature. Do not use if the nozzle or other parts are damaged. Do not hot sterilise. The product is destined for domestic use only. It is not recommended for use by immune depressed or immune compromised subjects. Report any serious accident to the manufacturer and to the competent authorities for the given country.

	Medical Device
	CE marking
	Product catalogue reference code
	Manufacturer
	Production Batch
	Expiring date
	Production date
	Instruction manual
	Unique Device Identifier
	Do not use if package is damaged

SOFT NOZZLES FOR NASAL ASPIRATOR

It contains 3/10 soft and delicate nozzles sealed in singular pack. Chicco allows to take care of babies' sensitive nose with the highest hygiene,

ne, offering individually packed nozzle. The nozzles in soft and flexible plastic are designed to perfectly fit for the delicate nose of the baby.

Disposable device (for single patient use only). Do not reuse: risk of possible infections and loss of functionality.

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Artsana S.p.A