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**INSTRUCTIONS FOR USE**  
**ThermoClinic**  
**Clinical Thermometer with Analog Device**

**CONTENTS** (Figure 1)

1 thermometer, 1 manual, 1 case with manual shake down device.

**TECHNICAL SPECIFICATIONS**  
Measuring range 35,0°C-42°C - Resolution: 0,1°C - Accuracy: +0,1°C/-0,15°C  
Operating Temperature: 10°C to 35°C - Storage Temperature: -10°C to 42°C.

**Quality guarantee**  
Standard EN 12470-1:2000+A1:2009 Clinical thermometers - Part 1: Metallic liquid-in-glass thermometers with maximum device.  
This thermometer without mercury, with liquid metal, is a clinical thermometer with an analogue device for measuring body temperature. Manufacturer is ISO 13485 certified. It complies with Medical Device EU Regulation 2017/745.

- WARNING**
- Do not drop the thermometer or subject it to sudden impact.
  - Do not bend or bite the tip of the thermometer.
  - Keep the thermometer out of reach of children.
  - Do not use the thermometer if the outer casing is damaged.
  - Do not use the thermometer in the ear.
  - Disinfect the thermometer before and after each use, especially if the device is used by multiple people.
  - Avoid high temperature storage, not more than 42°C.
  - Before use and test, please check the product, such as damage, liquid column disconnection, crack, liquid column can not move, paper falling off, do not use and test, to avoid affecting the product performance.
  - Children should be supervised by an adult when using it to prevent biting and swallowing.
  - Any serious incident related to the device must be reported to the manufacturer and the Competent Authority of the member state.

**INTENDED USE**  
The thermometer is intended to measure human body temperature via the armpit, orally, or rectally.

**INSTRUCTIONS FOR USE**  
The purpose of this device is to monitor patients' body temperature. The device can be used for clinical or home use on people of all ages.

- Before taking the patient's temperature, check whether the column of liquid is below the first numbered line (between 35°C and 36°C).
- Clean the glass thermometer before and after each use. Follow the precautions outlined in the paragraph on "Instructions for Cleaning".
- Position the thermometer according to the type of measurement to be taken (axillary, rectal or oral).
- After about 5 min the thermometer will have reached the body temperature. The temperature can be read by rotating the thermometer gently. The position of the fluid in the capillary allows for the body temperature to be read against the graduated scale.
- After the measurement, the liquid needs to be brought back down to below 36°C. Place the thermometer in the white shake down device and securely screw the shake down device in the plastic casing. Grasp the upper extremity of the shake down device and shake with 10 to 15 rapid wrist movements.

**WARNING:** Always disinfect the thermometer before use. In hospitals disinfection must be carried out by experienced personnel.

**Admissible methods for measuring temperature.**

**Oral**  
The bulb of the thermometer (probe) must be positioned under the tongue, keeping the mouth closed. The normal temperature using this measurement method is about 37°C (measuring time is 5 minutes).

**Rectal**  
The thermometer bulb (probe) is inserted into the anus. The normal temperature using this measurement method is about 37,6°C (measuring time is 5 minutes).

**WARNING:** Do not insert the tip of the probe more than 2 cm into the rectum; Stop immediately if there is any resistance. The operation can be facilitated by using some baby oil or cream. Do not use Vaseline.

**Armpit**  
Make sure that the armpit is dry. The tip of the thermometer must be positioned under the armpit and the arm pressed against the body. The normal temperature using this measurement method is about 36,7°C (measuring time is 7 to 10 minutes).

- WARNING**
- If used on children, an adult must be present for the entire duration of the temperature measurement process.
  - Be careful to avoid the danger of injury while taking measurements.
  - All of the thermometers have been tested by the manufacturer to ensure compliance with the prescribed tolerance limits (+0,1/-0,15°C).
  - Should the thermometer break, any liquid that spills out can be removed using a cloth dampened with soap and water. The shards of broken glass from the thermometer can be disposed of as domestic waste.

**CLEANING PRECAUTIONS**

- The thermometer consists of a hermetically sealed sanitary glass casing and can be disinfected without any restrictions.
- The cleaning of the thermometer before and after use can be done with soap and cold water.
- Disinfect the thermometer using a piece of cotton wool or cloth that has been soaked in alcohol or dip the thermometer in a disinfectant solution. The temperature of the disinfectant solution must not exceed 42°C.
- When cleaning, please take the necessary care to avoid any danger of injury.

**Notes on the use of the shake down device**

Place the thermometer in the white shake down device and securely screw the shake down device in the plastic casing. Grasp the white extremity of the shake down device firmly between the thumb

and index finger. Shake the shake down device at a high frequency; for this step do not shake your arm, only shake your hand. The shake down device needs to be shaken in rapid succession but not forcefully until the liquid drops below 36°C.

**LIMITED WARRANTY**  
Subject to normal use, this thermometer is guaranteed against any manufacturing defects for a period of five years from the date of purchase. Damage to the thermometer caused by improper treatment is not covered by the warranty.

**SYMBOLS**

|  |                                                     |  |                                                                         |
|--|-----------------------------------------------------|--|-------------------------------------------------------------------------|
|  | Follow the Instructions for Use                     |  | Temperature limits                                                      |
|  | Warning                                             |  | Keep in a dry place                                                     |
|  | Lot Number                                          |  | Fragile, handle with care                                               |
|  | Legal Manufacturer                                  |  | Medical Device                                                          |
|  | Glass thermometer without mercury                   |  | Distributor                                                             |
|  | Importer                                            |  | Unique Device Identifier                                                |
|  | Keep away from sunlight                             |  | CE conformity mark. 0197 represents notified body identification number |
|  | Authorized representative in the European Community |  | Manufacturing date                                                      |
|  | model number                                        |  | Product catalogue reference code                                        |
|  | Expiring date                                       |  |                                                                         |

**WASTE DISPOSAL**

Dispose according to local requirements.

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