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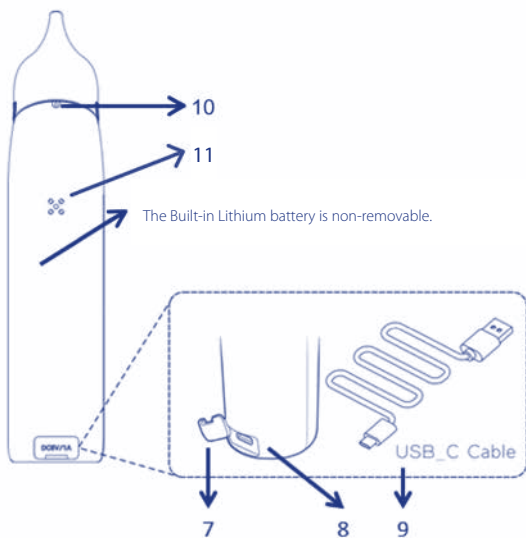
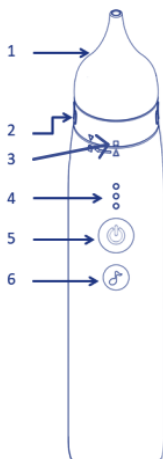
PICTURE A

USER MANUAL

Dear customer, thank you for choosing this Chicco Electric Nasal Aspirator. Before using the unit, we highly recommend that you read the shortlist of warnings contained in this manual, to make sure you have correctly understood how to use it.

IMPORTANT

Read the instructions carefully before use.

COMPONENTS (PICTURE A)

1. Silicone Nozzle - 2. Mucus Collection Cup - 3. Power Indicator Light - 4. Setting Indicator Light - 5. Power Button(On/OFF) - 6. Melody Button - 7. Plastic USB-Cover - 8. USB-C Charging Port - 9. USB-C cable - 10. Vent Hole - 11. Speaker for playing melodies.

INTENDED PURPOSE

The device is designed for using intermittent suction to remove nasal secretion and mucus in the home environment.

PATIENT POPULATION

Children (age 0~12 years old)

INTENDED USERS

The nasal aspirator is intended for use by laypersons, typically parents or caregivers, in a home setting. Two primary user requirements are defined as follows:

1. Knowledge – Minimum:
 - Ability to distinguish between the mouth, ear canal, and nasal cavity.
 - Basic understanding of hygiene principles.
2. Language comprehension:
 - Ability to read and understand the official local language for proper interpretation of the Instructions for use.

INDICATIONS

1. Proper aspiration of children's nasal mucus or secretions (recommended up to 4 times per day) can help alleviate upper and lower respiratory symptoms and associated conditions.
2. Regular and effective aspiration of a child's nasal mucus or secretions can help support respiratory health.

CONTRAINDICATIONS

Patients with damage to the integrity of the nasal mucous membrane or with specific anatomical variations of the nasal septum should not use the device.

WARNINGS

- Do not modify this device.
- Do not expose the device to extreme temperatures, high humidity, or direct sunlight.
- Avoid excessive force, shock, heavy impacts, or accidental drops.
- Charge the device when the power indicator begins to blink.
- The setting indicator light serves as both the battery charging status indicator when charging and the suction level indicator when operating.
- It is recommended to use the device intermittently. Press the power button to activate suction for removing nasal secretions. To stop suction, press the power button again.
- The device includes a safety feature: after operating continuously for one minute, it will automatically stop, to prevent overuse and potential injury to the nasal mucosa.
- The nasal aspirator is intended solely for cleaning/removing nasal secretions. Do not use the device for any other purpose.
- During use, parents should ensure that children do not hold their breath or swallow. The child's mouth should remain open to prevent ear pressure imbalance.
- If discomfort occurs while using the device, stop using it immediately and consult a physician.
- Comfort the child before and during suction to minimize emotional distress, which may affect the effectiveness of the procedure.
- Keep the device out of reach of children when not in use to prevent accidental ingestion of components.
- Avoid performing suction while the child is asleep, as it may disturb their sleep.
- Any serious incident related to the device must be reported to the manufacturer, local representative, and the competent authority of the Member State where the user is located.

Note:

- Under all circumstances, the device is intended only for the removal of nasal secretions.
- The device is in compliance with the European Regulation for medical device UE MDR 2017/745

HOW TO USE THE DEVICE

- Hold the baby or child in an upright position; the aspirator should not be used while the patient is lying down.
 - Take care not to insert the nozzle too deeply into the nostril or press it against the inner nasal wall.
1. Select the most appropriate nozzle based on the expected type of nasal secretions:

Short nozzle: Recommended for thicker or more solid mucus.

Long nozzle: Recommended for thinner or more liquid mucus.

2. Firmly attach the selected nozzle to the collection cup.
3. Ensure the collection cup is securely fastened to the device. The arrow on the collection cup must align with the lock icon on the device. (Twist the collection cup until you hear a "click" indicating it is locked in place.)

4. Press the power button to activate the aspirator and ensure it is functioning properly at the most correct suction level before inserting it into the nostril.
5. The power indicator light will illuminate while the device is in operation and also serves as a distraction light to capture the baby's attention.
6. Gently insert the silicone nozzle into the nostril, allow the suction for few seconds to remove the mucus and press the power button again to stop the aspirator.
7. Repeat in the other nostril as needed.

Note:

- Discontinue use immediately if any discomfort occurs.
- The device will operate for one minute and then stop automatically.
- Mucus may not be immediately visible in the collection cup. It is recommended to remove and rinse the silicone nozzle and collection cup between each use to prevent mucus from drying and clogging the device.

Suction Level Selection

The device offers three suction levels, with the initial default set to the lowest level upon first use. After the first use, the device will remember and revert to the last suction level selected when turned off.

- To adjust the suction level, press and hold the power button for 1.5 seconds. (Low → Medium → High)
- The setting indicator light displays the current suction level during operation:
 - o 1 light: Low suction
 - o 2 lights: Medium suction
 - o 3 lights: High suction

Before placing the device into the nostril, test the suction level on the back of your hand to determine the most suitable setting for the child. To adjust the suction level, it is advisable to withdraw the nasal aspirator from the nostril prior to changing the setting.

Music Selection

To comfort the child during suction and minimize emotional distress, you can play the built-in melodies.

Press the melody button to activate the melodies, press it again cyclically to change the melodies and choose the favorite one.

Press and hold the melody button for 1.5 seconds to stop the melodies.

CLEANING AND DISINFECTION

The following cleaning procedures should be performed daily, or as needed, when the device is in regular use. Failure to clean the unit properly may result in clogging and reduced product lifespan.

- Unscrew the collection cup from the device housing.
- Carefully detach the silicone nozzle from the collection cup.

- It is recommended to rinse the silicone nozzle and collection cup with hot water between uses to prevent mucus from drying and clogging the unit.
- Cleaning (daily/as needed): Wash the collection cup and silicone nozzle with mild, non-abrasive soap and water.
- Disinfecting (after illness/as needed): After cleaning, rinse the nozzle and collection cup with hot water at 70°C. The silicone nozzle can be disinfected using also boiling water or steam.
- Do not wash or immerse the main housing of the aspirator. Clean the device using a soft, dry cloth. Never use abrasive cleaning agents, thinners, or benzene.
- Do not expose the nasal aspirator to extreme temperatures, high humidity, direct

- sunlight, or mechanical shock.
- Allow all components to dry completely before reassembling.

Note:

Protect the nasal aspirator from mechanical shock or heavy impact. Store the unit in a location free from sunlight, dust, and humidity.

Additional Cleaning for Maintenance

To help prolong the life of the device, it is recommended to perform the following additional cleaning procedure when necessary:

1. Insert the device upside down into a container of clean water, ensuring that the water level does not exceed the collection cup.
2. Press the power button and operate the device until all dirty liquid is expelled.

3. After the liquid has been expelled, remove the device from the water and press the power button again to operate the device until all remaining water inside the pump has been fully out.

Perform this maintenance cleaning whenever a lot quantity of mucus is expelled from nostril or before storing the device for an extended period.

BATTERY RECHARGE

When the power light blink for 5 seconds the battery is low and the device will stop working. Use USB-C charging cable to charge the device.

For recharging the battery, please, use exclusively the cable provided with the product:

1. Connect the Type-C end of the USB cable to the device.
2. Connect the Type-A end of the USB cable to the power adapter (not included with the product).

3. The setting green indicator lights will blink while the battery is charging.
4. As the charge progresses, the number of steadily illuminated lights will increase, indicating the level of battery charge.
5. When the battery is fully charged, all three green indicator lights will be illuminated. Please unplug the charger at this point.

- The device will not charge if the battery temperature is outside the range of 5°C to 45°C. All three indicator lights will blink 5 times as a warning.
- Do not charge the device in environments with direct sunlight or high temperatures, as this may damage the battery and increase the risk of fire.
- It is recommended to recharge the battery with the device powered off.
- Disconnect the power source and cable once charging is complete to prevent overcharging and potential damage.
- Under extreme conditions, the battery may leak corrosive fluid. If this comes into contact with your eyes or skin, rinse immediately with water and seek medical attention.
- The built-in lithium battery is non-replaceable.
- Use a separate power supply 5V ≥1A, authorized by the respective National Certification Bodies (NCBs) that hold approval marks in accordance with IEC 60601-1.

MALFUNCTIONS and ACTIONS TO TAKE



PROBLEM	CAUSE	REMEDY
The suction is weak or absent.	Low Battery.	Recharge the device.
	The rubber O-ring on the collection cup housing is missing.	Make sure that the rubber O-ring on the collection cup housing is properly in place.
	The collection cup is not properly screwed in.	Make sure the arrow on the collection cup is aligned with the lock icon on the device.
The device is not charging properly.	The charging cable is not properly plugged in.	Remove the cable and plug it in again.
	Using a charger with incorrect specifications.	Make sure that the power supply output is 5V ≥1A. If the problem persists it is suggested to change power supply.
	The battery temperature is lower than 5°C or higher than 45°C.	Allow the battery temperature to return to the range of 5°C to 45°C.

TECHNICAL SPECIFICATIONS

ITEM	SPECIFICATION
Effective Vacuum Pressure	350~500 mmHg ($\pm 5\%$)
Sound Pressure Level	≤ 60 dB 30cm distance
Motor	3V/DC
Battery	3.7V, 500 mAh built-in Li-Po battery
Charging Input	5V / 1A
Nozzle	Soft Silicone
Operating environment	Temperature: 16°C~35°C (60.8°F~95°F) Relative humidity: 15%~85% (non condensing) Atmospheric pressure: 700~1060 hPa
Storage/Transportation environment	Temperature: -25°C~+55°C (-13°F~+131°F) Relative humidity: 15%~85% (non condensing) Atmospheric pressure: 700~1060 hPa
Weight	approx. 146g (with battery)
Dimensions	approx. 183mm x 62mm x 39mm (L x W x D)
Product Life Time	3 years

DEVICE SYMBOLS – MANUFACTURER'S DECLARATION AND GUIDELINES - ELECTRO-MAGNETIC EMISSIONS: SEE APPENDIX.

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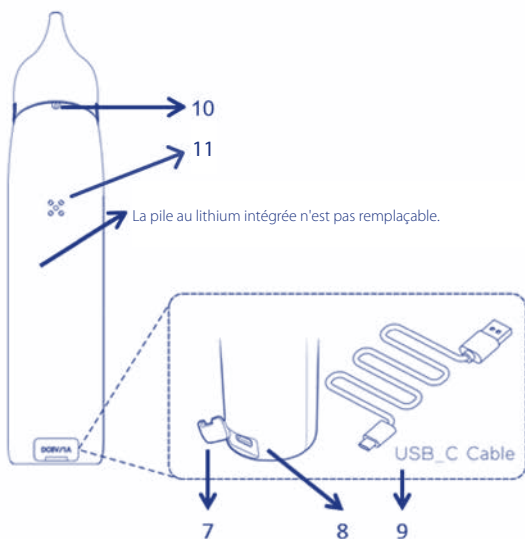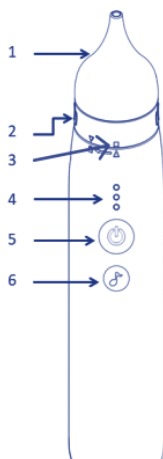EMC Tables

NS151 is intended for use in the electromagnetic environment specified below. The customer or the user of NS151 must make sure that it is used in such an environment.		
Guidance and manufacturer's declaration - Electromagnetic emissions		
Phenomenon	Professional healthcare facility environment a)	HOME HEALTHCARE ENVIRONMENT a)
Conducted and radiated RF EMISSIONS	CISPR 11 Group 1 Class A (Not BLE Function) Group 2 Class A (With BLE Function)	CISPR 11 Group 1 Class B (Not BLE Function) Group 2 Class B (With BLE Function)
Harmonic distortion	Not applicable (Note: Power by Battery or DC Input) Only the AC input needs to be tested	
Voltage fluctuations and flickering	Not applicable (Note: Power by Battery or DC Input) Only the AC input needs to be tested	
a) The equipment is suitable for use in Home Health Environments and Professional Health Care Environments limited to patient rooms and respiratory treatment facilities in hospital or clinics. The more restrictive acceptance limits of Group 1 Class B (CISPR 11) have been considered and applied. The equipment is suitable for use in the mentioned environments when directly connected to the Public Mains Network. b) The test is not applicable in this environment unless the ME EQUIPMENT and ME SYSTEM used will be connected to the PUBLIC MAINS NETWORK and the power input is otherwise within the scope of the Basic EMC standard.		
Guidance and manufacturer's declaration - Electromagnetic immunity - Enclosure port		
Phenomenon	Basic EMC standard or test method	Immunity test levels
		Professional healthcare facility environment HOME HEALTHCARE ENVIRONMENT
ELECTROSTATIC DISCHARGE	ELECTROSTATIC DISCHARGE	± 8kV contact ± 2 kV, ±4kV ±, ±8 kV, ±15 kV air
Radiated RF EM fields	IEC 61000-4-3	a)
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	COMPLIANT NOTE: Further information about distances to be maintained between portable and mobile RF communications equipment (transmitters) and the NS151 can be requested from supplier using the contact information provided in this manual. However, it is advisable to keep the equipment at an adequate distance of, at least, 0.5 m from mobile phones or other RF communications transmitters to minimise possible interference.
RATED power frequency magnetic fields.	IEC 61000-4-8	30 A/m c) 50 Hz or 60 Hz
a) The equipment is suitable for use in Home Health Environments and Professional Health Care Environments limited to patient rooms and respiratory treatment facilities in hospital or clinics. The more restrictive IMMUNITY acceptance limits have been considered and applied. b) Before modulation is applied. c) This test level assumes a minimum distance of at least 15 cm between the ME EQUIPMENT or ME SYSTEM and sources of power frequency magnetic fields.		

Explanation of Symbols

	Device Complies with European Regulation on Medical Devices EU MDR 2017/745
	Medical Device
	Manufacturer
	Authorized representative in the European Community
	Date of manufacture (YYYY-MM-DD or YYYY-MM)
	Batch code (YYMMWWWW)
	Serial number (YYMMWWXXXXX)

	Keep away from moisture
	Keep out of sunlight
	Temperature limit
	Humidity limitation
	Atmospheric pressure limitation
	Caution
	Consult the instruction for use
	Disposal information: Should you wish to dispose of the article, do so in accordance with current regulations. Details are available from your local authority. WEEE 2012/19/EU Directives
RoHS	This product fulfilling the requirements of the RoHS Directive 2011/65/EU.
REACH	This product fulfilling the requirements of the REACH Directive EC 1907/2006 and its amendments, do not contain Substances of Very High Concern in concentration above the limit of 0.1 %. No substance(s) is/ are present in the parts of the product above the concentration of 0.1 % weight by weight.
	Device classification type BF
IP22	This product meets the basic safety and essential performance requirements indicated in the IP22 conditioning test (protection against solid foreign objects of 12.5mm and greater and against vertically falling water drops when enclosure tilted up to 15°)
	The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
	Importer
	Distributor
	Model Number
	Country of Manufacturer
	Unique Device Identifier
	Catalogue number

[illegible]

This image shows a single sheet of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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Model: PHYSIOCLEAN ELECTRIC NASAL ASPIRATOR # NS151

REF 00012585000000

AVITA Corporation
9F., No. 78, Sec. 1, Kwang Fu Rd.,
San Chung Dist., New Taipei City 24158, Taiwan, China

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MDSS GmbH
Artsana S.p.A
MDSS CH GmbH