PHYSIOFORMA® GOMMOTTO PREMATURI IN SILICONE

PER SOLO USO
OSPEDALIERO

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EN PRETERM SILICONE SOOTHER, SUITABLE FOR NEWBORNS WHO WEIGH LESS THAN 2,000 g - READ THESE INSTRUCTIONS AND KEEP FOR FUTURE REFERENCE.

FOR THE CHILD'S SAFETY: WARNING!

⚠ Sooter for preterm babies, exclusively for hospital use, must not be used by newborns who weigh more than 2.000 grams. Any other use is to be deemed improper and potentially dangerous. This soother is classified as a class I medical device in compliance with the requirements set forth in Regulation 2017/745 of the European Parliament. Stimulates and encourages nutritional soothing. Parents must NOT use this soother at home as usage must always be carefully evaluated by specialised medical and/or nursing personnel and implemented under suitable surveillance. Due to the specific use for which this product was designed, the dimensions of the product are quite small and are not therefore compliant with the technical standard EN1400 for normal soothers. Inspect the soother before each use and pull the feeding test in all directions. Throw away at the first signs of damage or weakness. Never attach to cords, ribbons, laces or loose parts of clothing. Risk of strangulation. Do not leave a feeding test in direct sunlight or heat. Do not leave the soother in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the material.

HYGIENIC MEASURES: Before first use, place the soother in boiling water for 5 minutes and then allow it to cool. The purpose of this step is to ensure hygiene. Rinsing and sterilising the soother in water before each use is recommended (do not wash in a dishwasher). The soother may be sterilised using cold water sterilising solutions or by heat sterilisation (steam) using appropriate devices or in an autoclave with 20 minute cycles at 121 °C or 134 °C, up to a maximum of 30 sterilisations. After washing and sterilising free the teat from any residual liquid. For hygiene and safety reasons each soother should be used by one baby only and must be replaced after a maximum of 1 month of use; stop using if the baby's weight rises above 2.000 grams. When not in use, store the product in a closed container that is clean and dry. Keep the transparent protective bag out of reach of children. Store the product in a cool, dry place. Report any serious accident to the manufacturer and to the competent authorities for the given country. Preterm soother for hospital use only - Sooter may not be sold singularly.

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Artsana S.p.A. - Via Saldarini Catelli, 1