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BREAST SHELLS

CONCHAS RECOGELECHE

CONCHAS COLECTORAS DE LEITE

COQUILLES

MILCHAUFFANGSCHALEN

MELKOPVANGSCHELPEN

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EN BREAST SHELLS

Please read these instructions carefully and keep for future reference.

BREAST SHELLS offers a positive help during the delicate period of breast-feeding.

Collects milk spills during feeding, preventing prolonged skin contact with milk residues and offering protection for clothing.

Allows nipples to breathe thanks to the ventilation holes, which permit air to circulate and keep skin dry. They also help to heal nipple irritation or cuts.

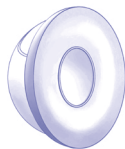
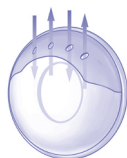
Relieves breast congestion, by exercising a delicate pressure around the nipple area, helping also to stimulate the flow of excess milk.

Stimulates the extension of **flat or retracted nipples**, thanks to their delicate massaging action around the nipple area.

The **soft silicone membrane** provides comfortable skin contact and gently adapts to the breast's shape.

The **anatomical design** fits the breast's shape.

The **supplied beaker** allows the easy pouring of milk.

CLEANING AND HYGIENE

1. Before using **CHICCO BREAST SHELLS** for the first time, separate each component and wash thoroughly with water and domestic washing up liquid, then rinse carefully.
2. Before every use, sanitise the product (after having separated all of the parts) in boiling water and/or with an electric steriliser, a steriliser for the oven or the microwave oven, or cold water sterilising equipment.
3. After use, wash the product carefully as indicated at point 1, taking care to eliminate all traces of milk residue. After drying, store in a clean, dry place, away from sunlight or direct heat sources. Do not wash in a dishwasher.

HOW TO USE CHICCO BREAST SHELLS to Collect Excess Milk

Wash hands and breast carefully.

Place the soft silicone membrane over the breast shell taking care that it attaches firmly. Place the breast shell inside the bra and rest the nipple inside the opening, ensuring that the ventilation holes are facing upward. Ensure that the shells do not exert too much pressure on the breast when wearing tight clothing.

To empty the shell, gently remove the silicone membrane.

If you wish to reuse the milk during feeding, transfer it immediately into a sterilised bottle from the Chicco range. After removing the **BREAST SHELLS**, carefully wash your breast to eliminate all milk residues.

In Cases of Flat or Retracted Nipples.

CHICCO **BREAST SHELLS** also help mothers with flat or retracted nipples to breast-feed their baby. The soft silicone membrane applies a light but continuous pressure on the areole and gently pushes the nipple outward; this allows baby to adapt to the correct position while feeding on the mother's breast. For information on hygiene and how to use the shells, follow the instruction at points 1, 2 and 3. We recommend using Breast Shells during the night and always in a lying down position. In this case, you can use the single use CHICCO PADS.

WARNING

- When not in use, keep the product out of reach of children.
- Do not use the product when showing signs of damage or wear.
- When not in use, store the product in a clean, dry, closed container, away from sunlight or direct heat sources.
- Keep all packaging (such as plastic bags, cardboard boxes. etc) out of children's reach, as they could present a source of danger.
- This product is expressly intended only for the use that it was designed for (i.e. as Breast Shell).
- Report any serious accident to the manufacturer and to the competent authorities for the given country.

Can be cold sanitised

Can be hot sanitised

	CE marking
	Consult the instruction manual
	Warning
	Product catalogue reference code
	Manufacturer
	Production Batch
	Medical Equipment
	Production date

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Artsana S.p.A. - Via Saldarini Catelli 1 - 22070 Grandate (CO) Italy