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Before using this product, please read these instructions and retain for future reference.

- Easy to use.
- The cup is made from heavy-duty, hygienic, plastic material that is easy to sterilise.

CLEANING AND DISINFECTING
Remove the suction pump (1) from the plastic cup (2). Carefully was all the components to remove all milk residue. Plastic cup: cold sanitise, using specific sterilising solutions and/or hot sanitise (steam) it using relative Chicco appliances or simply boil it.

Suction pump: use a soft cloth to clean it.

USER INSTRUCTIONS
The Chicco Nipple Corrector should be used just before you start

every feed.
Wash it throughly before use.
Wash your hands and cleanse your breasts thoroughly. Press and hold the suction ball and place the plastic cup on the nipple, checking it is positioned correctly on the areola.

Now gently release your hold on the ball. Repeat this procedure until the nipple is fully extended.

The baby can now be latched onto the breast and will start to suckle. Cleanse the breasts thoroughly after each breast feed.

WARNINGS
• Keep the product out of children.

- Do not use the product if any parts are broken or missing.
- Do not leave the nipple corrector exposed to direct sunlight or heat sources. Store in a closed, clean container and keep in a cool, dry place.
- Do not leave any packaging materials (plastic bags, cardboard boxes, etc.) within the reach of children as they can be hazardous.
- This product is only intended for the use it was specifically designed for: that is a nipple corrector.

	Consult the instruction manual
	Warning

	Product catalogue reference code
---	----------------------------------

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