Tiralatte Classico

Classic Breast Pump

Sacaleche Clásico

Saca-Leite Clássico

Hand-Milchpumpe

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Klassieke Borstkolf

Klassisk Mjölkpump

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Classic Breast Pump

Please read these instructions carefully before use and keep them for future reference.

The Classic Breast Pump provides maximum safety during this delicate nursing period, and is ideal for all situations where mums cannot feed their baby themselves.

It is easy to use and extracts the milk quickly and efficiently.

It has a suction rate control system which makes the pump work faster or slower as desired: this guarantees gentle and delicate extraction without painful tugging on the nipples.

The device comprises just a few parts in hygienic plastic which are very easy to sterilise. If you wish to feed baby immediately, fit the ring and the Well-Being teat, included in the package, to the bottle. You will then have a handy ready-to-use Well-Being feeding bottle.

WARNINGS

- Keep the breast pump and its accessories out of reach of children.
- Before use, ensure that the breast pump and its components are not worn or damaged. In the case of doubt, do not use and contact one of Artsana's authorised retailers.
- Do not leave any packaging materials (plastic bags, cardboard boxes, etc.) within the reach of children to prevent dangers.
- This product is only intended for the use it has been specifically designed for: it is a breast pump/feeding bottle. All other use is to be considered improper.

LIST OF COMPONENTS

1. Suction ball breast pump
2. Pressure relief valve
3. Connector pipe
4. Cup
5. Cap ring
6. Silicone seal
7. Well-Being teat
8. Well-being feeding bottle - 150ML
9. Base
10. Cap

Before using the breast pump for the first time, remove, wash and carefully sanitise all components (except the suction ball pump (1) and the pressure relief button (2)) as illustrated in the **CLEANING, SANITISING & MAINTENANCE** section.

CLEANING SANITISING & MAINTENANCE

Before extracting the milk, cold sanitise all the components, except the suction ball pump (1) and the pressure relief valve (2), using suitable sterilising solutions, and/or hot sanitise (steam) them using relative appliances, or you can also boil them.

To clean the suction ball and pressure relief valve, use a damp sponge (with a couple of drops of washing up liquid if necessary) and dry with a soft cloth.

Store in a clean, dry place away from direct light and all heat sources.

USING THE BREAST PUMP

- insert the silicone seal (6) in the cup(4).
- Screw the cup (4) to the bottle (8), checking if the silicone seal (6) is placed correctly
- Insert one end of the pipe (3) in the cup (4) and the other end in the valve (2) . Insert the valve inside the ball pump (1).
- Wash your hands thoroughly and cleanse your breasts using Chicco Sure-Safe nursing cleansing wipes .
- Position the breast pump applicator firmly on the breast.
- Lean forwards slightly to stimulate the milk flow.
- Press the suction ball gently at regular intervals (1).
- If the suction rate is too strong, press the pressure relief button on the valve to regulate it as desired (2).
- Alternate the extraction of milk from one breast to another, until they feel completely empty.
- To feed the baby immediately, fit the cap ring (5) and the Chicco Well-Being teat (6) to the feeding bottle (7). You will then have a handy ready-to-use feeding bottle.

By closing the bottle with its airtight cap (9), the milk can be stored in the fridge (max 72 hours) or in the freezer (max 3 months) whilst preserving all its natural nourishing ingredients.

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Artsana S.p.A. - Via Saldarini Catelli 1 - 22070 Grandate (CO) Italy