46 005217 000 000\_01\_2019  
Istruzioni d'uso Rev. 00 del 16-05-16 (Etichetta 3c-iso rev. 00 del 16-05-16)


TOSVAR S.r.L. - Via del Lavoro 10  
20060 Pozzo D'Adda (MI)  
Tel 0290960289 - info@tosvar.it

Distribuito da - Distributed by - Distribuído por - Διανέμεται από:  
Artsana S.p.A. - Via Saldarini Catelli 1, 22070 Grandate (Como) - Italy  
www.chicco.com

Made in Italy, Made in Italy, Fabricado en Italia,  
Κατασκευάζεται στην Ιταλία, Italy'da üretilmiştir

CE  
0546

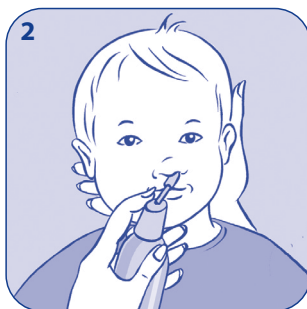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Numero verde consumatori Italia 800 188 898

## EN INSTRUCTIONS FOR USE ISOTONIC SEA WATER PHYSIOCLEAN SPRAY

- Easy to use multi-position spray, thanks to the 360° technology it can be used in all positions to make the nasal wash more complete.
- Cleanses and clears the nose, helping to eliminate excess mucus.
- Naturally moisturizes and protects the nasal mucous membranes.
- It gives relief to irritated or dry mucous membranes.

- Also indicated in case of allergies or colds.
  - Extra-delicate spray suitable for children.
  - Special "Comfort Stop" dispenser with safety edge to avoid excessive insertion. Essential to use in children from 6 months to 2 years.
- Preservative free. Tested tolerance.

**Contains:** Sea salt, Water, Nitrogen.



### Instructions:

1. Bring the nostril closer to the

- bottle (fig.1).
  2. Gently insert the dispenser into the nostril and exert pressure on the dispenser (fig.2).
  3. Repeat the operation with the other nostril (fig.3). Dispense once in each nostril one to four times a day.
  4. Remove secretions with a handkerchief.
  5. Rinse the dispenser with hot water after use. Suitable for daily use.
- 1 to 4 deliveries per day are recommended.

**Warnings:** Not indicated for use

in children under 6 months. Do not use after the expiry date specified on spray can. Follow information for use. Pressure container: it may burst if heated up. Keep it away from heat sources, hot surfaces, sparks, open flame or other ignition sources. Do not smoke. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures above 50°C/122°F. Keep out of reach of children. Do not spray on open flame or other ignition source. Do not spray into eyes.

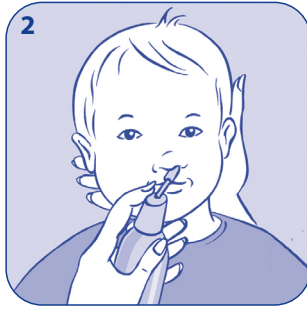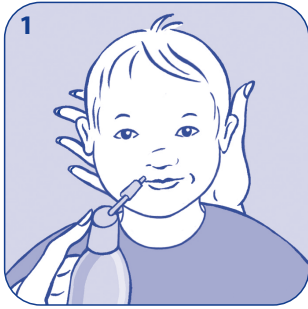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