TIRALATTE MANUALE

BREAST PUMP

SACALECHE MANUAL

SACA-LEITE MANUAL

MILCHPUMPE

TIRE-LAIT MANUEL

HANDMATIGE BORSTKOLF

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Please read these instructions carefully before use and keep them for future reference.

LIST OF COMPONENTS

1. Soft silicone membrane
2. Anatomical cup
3. Ergonomic handle
4. Flow regulation valve
5. One-way valve
6. Step UP connector
7. Well-Being connector
8. Step Up 1 feeding bottle – 150ml
9. Step Up 1 teat
10. Cap ring
11. Cap
12. Holder base

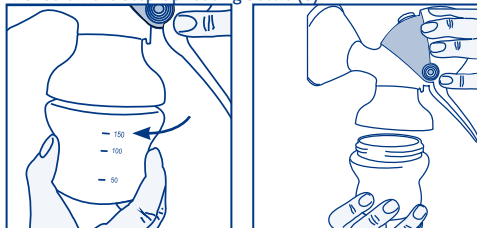
Before using the breast pump for the first time, remove, wash and carefully sanitise all components (except the handle (3)) as illustrated in the **CLEANING, SANITISING & MAINTENANCE** chapter.

CLEANING SANITISING & MAINTENANCE


Before extracting your milk, cold sanitise all the components, **except the handle (3)**, using suitable sterilising solutions, and/or hot sanitise (steam) them using relative appliances; alternatively you can also boil them.

DISASSEMBLING THE BREAST PUMP

1. Unscrew the Step Up feeding bottle (8)

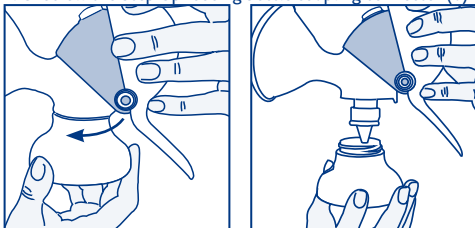
2. Unscrew the Step Up feeding bottle coupling connector (6)


Using the Well-Being feeding bottle (not included in the pack)

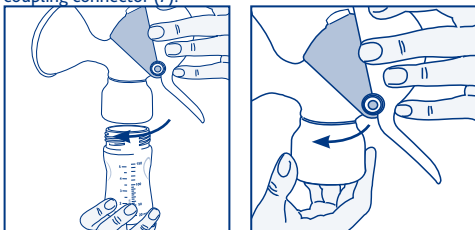Unscrew the Well-Being feeding bottle (not included) and then the Well-Being coupling connector (7).

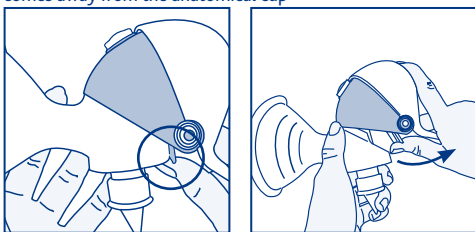
3. Release the handle tab from the cup seat and pull upwards until the handle comes away from the anatomical cup


Warning: The handle (3) is the only component that never comes into with the milk; it is therefore sufficient to clean it with a soft cloth. If the handle is rinsed, dipped into water, cold sanitised with sterilising solutions, hot (steam) sanitised in sterilisers, or boiled this can prevent the product from working properly. It is highly recommended to maintain the handle periodically, following the instructions provided in the **Handle Cleaning and Maintenance** section.

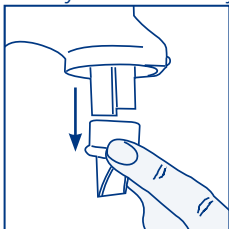4. Gently remove the one-way valve (5)

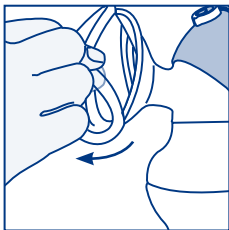5. Remove the silicone membrane (1)

After use, always wash the breast pump cup (2) thoroughly to eliminate any milk residue.

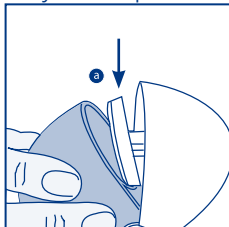
HANDLE CLEANING & MAINTENANCE

The handle (3) is the only component that never comes into with the milk; it is therefore sufficient to clean it with a soft cloth.

It is recommended to oil the piston ring regularly or when necessary should the handle become stiff or difficult to move; use a small amount of baby oil or a similar product. This operation prevents wear and aging of the ring. Before starting maintenance, remove the handle (3) from the anatomical cup (2) as illustrated in the **Disassembling the breast pump** section.


Push the handgrip until you can see the piston ring (a) and moisten it with a little oil.

Always store the product in a clean, dry place.


ASSEMBLING THE BREAST PUMP

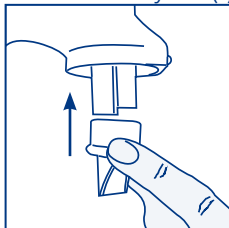
1. Insert the one-way valve (5) as far as possible into the anatomical cup seat.

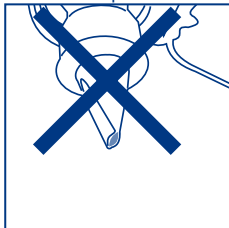
Warning. To ensure the correct operation of the Breast Pump, always check that the one-way valve is accurately fitted as far as possible into its seat.


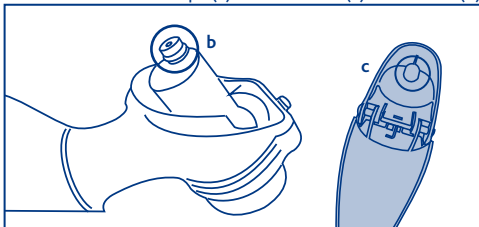
2. Insert the anatomical cup's (2) rod and washer (b) into the seat (C) on the handle (3).

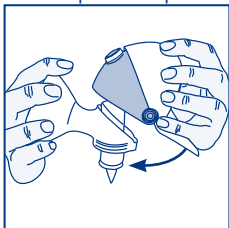
3. To assemble it correctly, apply light pressure downwards until you hear the handle flap click into place

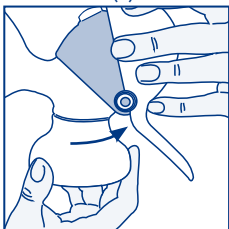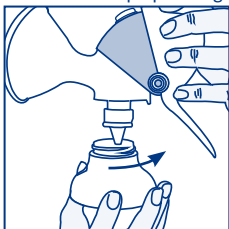4. Screw the Step Up feeding bottle connector (6) onto the anatomical cup (2).

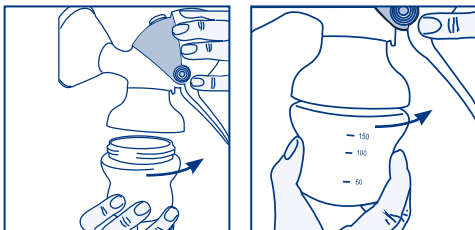
5. Now screw on the Step Up feeding bottle (8)


Using the Well-Being feeding bottle (not included in the pack)

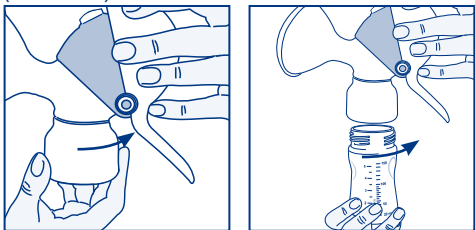Screw on the Well-Being connector (7) and then the Well-Being feeding bottle (not included).

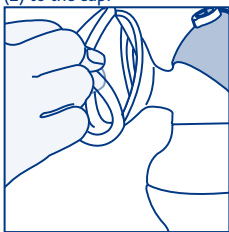6. For maximum comfort, it is recommended to apply the soft silicone membrane (2) to the cup.


HOW TO USE THE MANUAL BREAST PUMP

Wash your breasts in soap and water or, alternatively, use the practical CHICCO Natural Feeling cleansing wipes. It is not necessary to cleanse the breasts if you are using the **Chicco Natural Feeling Anti-Bacteria Nursing Pads**. Sit in a comfortable position and relax. It can be useful to gently massage your breast with concentric movements of your fingers, starting from the outer area and moving in towards the areola; this will stimulate the galactophore ducts and help the milk to flow more easily.


Position the anatomical cup (2) comfortably on the breast.

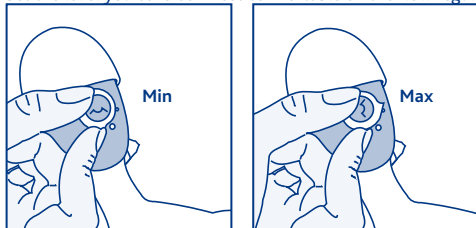Set the level you consider most comfortable on the flow regulation valve (4).

If the breast pump is being used for the first time, we recommend to set the valve to the minimum level and to increase it gradually until the most suitable level of intensity is found. In case the suction intensity selected causes an irritating sensation, it is recommended to reduce the intensity.

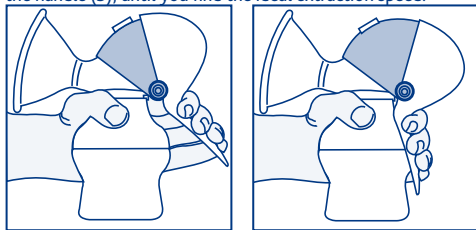It is possible to adjust the suction speed by applying gentle, rhythmic pressure to the handle (3), until you find the ideal extraction speed.

Warning: If the movement of the handle is difficult or insufficiently smooth, it is recommended to slightly oil the piston ring with some baby oil or similar product (to perform this operation, follow the instructions provided in the **CLEANING, SANITISING & MAINTENANCE** section. And **Handle Cleaning & Maintenance** section).

You can alternate the extraction of milk from one breast to another, until you feel they are completely empty.

If you wish to feed baby immediately, fit the cap ring and the Step Up 1 teat included in the package, to the bottle. You will then have a practical Step Up 1 feeding bottle ready for use.

WARNINGS

- Keep the breast pump and its accessories out of reach of children.
- Before use, ensure that the breast pump and its components are not worn or damaged. In the case of doubt, do not use and contact one of Artsana's authorised retailers.
- Do not leave any packaging materials (plastic bags, cardboard boxes, etc.) within the reach of children to prevent dangers.
- This product is only intended for the use it has been specifically designed for: it is a breast pump/feeding bottle. All other use is to be considered improper.

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Artsana S.p.A. - Via Saldarini Catelli 1 - 22070 Grandate (CO) Italy