Tiralatte Manuale NaturalFeeling

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EN NaturalFeeling Manual Breast Pump

Breastfeeding is a very important stage in a baby's and in a mother's life. For babies, latching on and sucking is not just satisfying a primary physiological need, it is also, and more importantly, a precious moment in the relationship with their mothers, full of emotions and tactile, auditory, gustatory, and olfactory sensations. Satisfying this need gives a great feeling of tranquillity and wellbeing to both the mother and the child. Breast milk contains all the nutrients required to meet infants' nutritional and psychological needs at least for the first 6 months of their lives.

Chicco's manual breast pump helps extend breastfeeding in the event of excess milk or when you have to be away from baby. In such cases you can express milk, store it, and give it to baby when necessary.

LIST OF COMPONENTS [Fig. 1]

- 1 Diaphragm
- 2 Piston
- 3 Handle
- 4 Soft silicone cushion
- 5 Pump body
- 6 NaturalFeeling Adapter (included in the packaging of code 00 005740 000 XXX)
- 7 Valve
- 8 NaturalFeeling Bottle (included in the packaging of code 00 005740 000 XXX)
- 9 Well-Being Bottle (included in the packaging of code 00 005740 100 XXX)
- 10 Base

WARNINGS

- Keep the appliance and its accessories out of reach of children.
- Do not leave any packaging materials (plastic bags, cardboard boxes, etc.) within the reach of children as these can be hazardous.
- Before every use, make sure that the appliance and its components are intact with no visible damage. If in doubt, do not use the appliance and contact qualified personnel.
- This appliance is intended for the sole use it has been specifically designed for. Any other use is to be considered improper.
- This product must be used solely with bottles of the Chicco Well-Being and NaturalFeeling ranges. Do not use any other bottles. To use and clean the bottle and its accessories, refer to the specific instructions supplied with the breast pump.

PREPARATION AND CLEANING

- Disassemble all the parts of the breast pump (see Fig. 1). Take care when removing and cleaning the valve. Remove the valve by gently pulling the tab on the side of the valve. Do not use tools to clean the valve. If damaged, the breast pump will not operate correctly.
- Wash every part thoroughly with water and washing up liquid. Rinse thoroughly and dry carefully using a soft, dry cloth or leave to dry in a clean and ventilated place. Do not use abrasive products to clean the breast pump.
- All the components, except for the valve and the handle, can be washed in the dishwasher using a delicate program. An aggressive type of washing may cause the material to become discoloured and dull over time.
- Disassemble the appliance completely before every use and sanitise the parts using cold or hot (steam) sterilising liquids and sterilisers, or boiling them for 5 minutes. Caution: leave the components to cool down completely before every use. Do not cold sanitise the handle (sterilising liquids).
- Disassemble the product right after every use and rinse every part of the breast pump thoroughly with hot water. Dry carefully using a soft, clean cloth or leave to dry in a clean and properly ventilated place.
- Store the product in a clean, dry place.

ASSEMBLING THE BREAST PUMP [Fig. 2-3-4-5-6-7]

- Insert the silicone valve on the round duct on the pump body (Fig. 2). Push the outer edge to prevent damaging the valve and make sure that both edges of the assembled valve are joined together.
- Assemble the silicone diaphragm on the piston, taking care that the reference ribs on the components correspond (Fig. 3).
- Insert the (pre-assembled) piston and diaphragm on the pump body. Make sure that the protruding edges of the diaphragm match with their housings visible on both sides of the pump body (Fig. 4) and make sure that the arrows on the pump body and the diaphragm correspond. Ensure that the diaphragm is positioned correctly and adheres to the body by pressing along the rim of the diaphragm with your fingers.
- Attach the handle to the piston, engaging the opening of the handle with the top of the piston (Fig. 5/1). Push the handle towards the pump body (Fig. 5/2). If it is assembled correctly, you will hear it click into place.
- For greater comfort, you can assemble the soft silicone cushion on the pump body (Fig. 6). Push the silicone cushion inside the pump body as far as possible and seal it around the outer rim of the pump body. Press the bearings to remove any air trapped inside.

- Assembling the bottle:
 - The bottles of the Well-Being range do not require the use of the adapter.
 - As for the bottles of the NaturalFeeling range, tighten the adapter on the pump body (Fig. 7/1) and then tighten the bottle to the adapter (Fig. 7/2).
- Use the practical base to lay the assembled product on a flat surface and improve its stability.

USING THE BREAST PUMP

- Wash your hands and cleanse your breast with water or using one of the practical Chicco breast cleansing wipes.
- To accommodate the milk flow, you can take a hot shower or apply a warm cloth on your breast just before expressing milk or breastfeeding your child.
- Sit down comfortably and relax.
- Fit the cup over your breast ensuring that the nipple is centred properly. You can use the breast pump both with and without the soft silicone cushion. We recommend trying both configurations before choosing the best and most comfortable one for you.
- Hold the handle down gently until you feel the suction on your breast, then release it. You can adjust the rhythm according to the traction felt on your breast or to the amount of expressed milk. You may need to push the handle all the way down.
- Start with quick movements, holding down and releasing the handle to stimulate milk expression. Once the milk starts flowing, slow down the movements by holding down the handle for 1 or 2 seconds before releasing it.
- You can express milk from both breasts alternatively.
- You can feed your baby with the breast milk right away by applying the ring and teat to the bottle. This way, you will obtain a handy baby bottle (Well-Being or NaturalFeeling) ready for use.
- Close the bottle with the airtight cap (included in the packaging) to store the expressed milk in the fridge for up to 48 hours or in the freezer for up to 3 months.

HOW TO USE THE BOTTLE

READ THESE INSTRUCTIONS AND KEEP FOR FUTURE REFERENCE FOR YOUR CHILD'S SAFETY AND HEALTH

WARNING!

- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Throw away at the first signs of damage or weakness.
- Keep components not in use out of the reach of children.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- Never use feeding teats as a soother.
- Always use this product with adult supervision.
- Glass containers may break.

INSTRUCTIONS FOR USE

- Before first use, disassemble and clean the product and then place the components in boiling water for 5 min. This is to ensure hygiene.
- Before each subsequent use wash carefully with hot water and washing up liquid, then rinse thoroughly to ensure hygiene.
- Make sure that the product is assembled correctly before use to ensure the correct functioning of the feeding bottle and to prevent any leaks.
- Keep the product upright at all times with the cup inserted during transport.
- Inspect the feeding teat before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness. In any case, we recommend replacing teats at least once every two months.
- To ensure the teat is functioning properly, we recommend gently massaging the valve at the base of the nipple with your fingers before each use. Carry out the manoeuvre gently, taking care not to damage the valve.
- Inspect the product before every use, making sure there are no broken parts or sharp edges. Throw away at the first signs of wear or damage.
- Do not leave a feeding teat in direct sunlight or heat. Do not leave in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the material.
- Ensure that the flow meets the baby's requirements, paying attention to any sucking problems or to milk coming out from the baby's mouth.
- Follow the pediatrician's recommendations for proper nutrition.
- When reconstituting powdered milk, the manufacturer's instructions must be strictly adhered to.
- Do not allow your baby to run or play with the product.
- Glass bottles must not be held by children.
- Use only genuine components and spare parts from the Chicco NaturalFeeling range.
- Remember that breastfeeding is the best for your baby.

CLEANING

- Wash all parts by hand with warm water and a regular washing-up liquid. Washing it in the dishwasher may cause the product to wear more quickly. Always rinse all parts thoroughly after each wash.
- Avoid using abrasive products or washing with aggressive chemical solvents. If bottle brushes are used, make sure you do not damage the more delicate parts of the product (e.g. teat openings and valves).
- Thoroughly clean under running water to eliminate any residual liquid that can nest in hidden areas.
- Sterilise before use.
- Ensure no residual liquid remains in the teat after washing or sterilisation.
- After cleaning, thoroughly dry all parts and store in a clean, dry place away from direct sunlight and sources of heat.
- The pigmentation of strongly-coloured drinks (e.g. tea, fruit or tomato juices) may alter the colour of the product. This does not alter the function and/or safety of the product.

ASSEMBLY

- Make sure that the product is assembled correctly before use to ensure proper bottle operation and prevent any leaks.

STERILISATION

- Before steam sterilising or sterilising in boiling water, the feeding bottle must be disassembled completely in order to prevent it from deforming and to ensure that boiling and sterilisation are effective.
- The product can be sterilised using cold sterilisation solutions, steam sterilised using suitable equipment or sterilised by boiling for 5 minutes, avoiding contact with the metal walls of the receptacle used.
- UV sterilisers are not recommended. Sterilising with UV sterilisers may cause the product to wear more quickly.

WARMING THE BOTTLE

- To warm the content, we recommend using a bottle warmer after removing the cap and referring to the bottle warmer instruction manual.
- Take extra care when microwave heating the feeding bottle:
 - Remove the cap, the ring and the teat before placing the bottle in the microwave oven: risk of explosion.
 - If you use a combination microwave oven, ensure that the grill is off.
 - Localized over heating can occur: always mix the heated liquid to ensure even heat distribution and check the temperature before feeding to prevent scalding.
- Do not heat the product in traditional ovens or on hot plates or open flames.

Plastic baby bottles: Can be placed in the freezer. Remove the teat and use the appropriate cap for storage.

- Pour the milk in the container taking care not to exceed the maximum capacity indicated on the graduated scale; tighten the airtight cap and store the container in the fridge or in the freezer.
- The milk can be stored for 48 hours in the fridge and for 3 months in the freezer. We recommend storing the milk in the fridge or in the freezer immediately after expression.
- Do not refreeze thawed breast milk.
- Do not add fresh breast milk to frozen breast milk.
- Remove the cap and/or the non-spill cup from the container when heating the milk or any other food and/or beverage. Do not leave the container in a hot water bath or in the bottle heater with the water boiling.
- You can heat the container in a microwave by removing the teat, ring, cup, and cap.

TROUBLESHOOTING

Problem	Cause	Solution
The breast pump is not pumping or is not pumping enough	Valve or diaphragm not assembled or badly assembled	Ensure that the valve and the diaphragm are assembled correctly
	The breast does not fit in the pump body	Position the breast pump better for the breast to fit properly and ensure that the nipple is centred. Try with/without the soft silicone cushion
Painful expression	The suction mode is too high	The suction mode depends on how you press the handle and on the speed of the movements of the grip. Change the speed and the way you hold down the handle to find greater comfort during stimulation and expression.

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Artsana S.p.A. - Via Saldarini Catelli 1 - 22070 Grandate (CO) Italy