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EN NIPPLE SHIELDS
■ PLEASE READ BEFORE USE AND KEEP THESE INSTRUCTIONS FOR FUTURE REFERENCE
Nipple shields are for personal use only.
HOW TO USE CHICCO NIPPLE SHIELDS
Wash your hands well and clean your breast thoroughly. Before use, wet the product in lukewarm water for better adherence and to favour the transmission of natural warmth. Fold back the edges and place the teat of the device carefully on your nipple; spread out the shield delicately and make it adhere well to the breast (Figures A and B). Make sure that the shape of the nipple shield matches the position of your baby's nose while feeding and that the nipple shield is not in contact with the areola. Positioning your baby correctly on your breast will ensure that your baby latches correctly on your breast taking both the nipple and areola. If feeding, remove the nipple shield.

WARNING:
- Warning: This product is not a toy; keep out of children's reach.
- Check the nipple shield before each use; do not use the product if it shows signs of wear or damage.

- Do not leave the nipple shield in sterilizing solutions for longer than recommended, as this may damage it.
- Keep the product out of reach of children when it is not in use.
- Never expose the nipple shield to sunlight or heat sources. Keep it in its box.
- Do not leave any packaging materials (plastic bags, cardboard boxes, etc.) within the reach of children as these can be hazardous.

WARNING:
If your baby's first teeth have come out, the silicone may be torn during sucking.

CLEANING AND STERILISING
Before using Chicco nipple shields the first time, boil them for at least 5 minutes and then leave them to cool.

Before each use sterilise the nipple shields by boiling and/or electric steriliser, or cold-sterilise using a Chicco broad spectrum disinfectant. The package can also be sterilised using the same methods. After every use, wash each part of the nipple shield carefully by hand with a common detergent for plates, then dry it and store it in its box. Do not leave the nipple shield in the microwave oven in its package in a clean, dry place away from light and direct heat sources. The nipple shields can be sterilised in autoclave: steam at a temperature of 121 °C for 15 minutes. The package and nipple shields, in addition to the methods specified above, can be sterilised directly in the microwave oven at 1000-750W for 3 minutes. - Remove the nipple shield from the package and place it in the microwave oven. - Set the microwave to 1,000-750W; cycle time 3 minutes.

Attention: at the end of the cycle, the package and nipple shields may be very hot. Danger of scalding. Wait a few minutes and leave to cool before using.

Do not exceed the recommended cycle time to avoid making the water evaporate and hence damaging the product.

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