Super Rino

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REF 000906600000

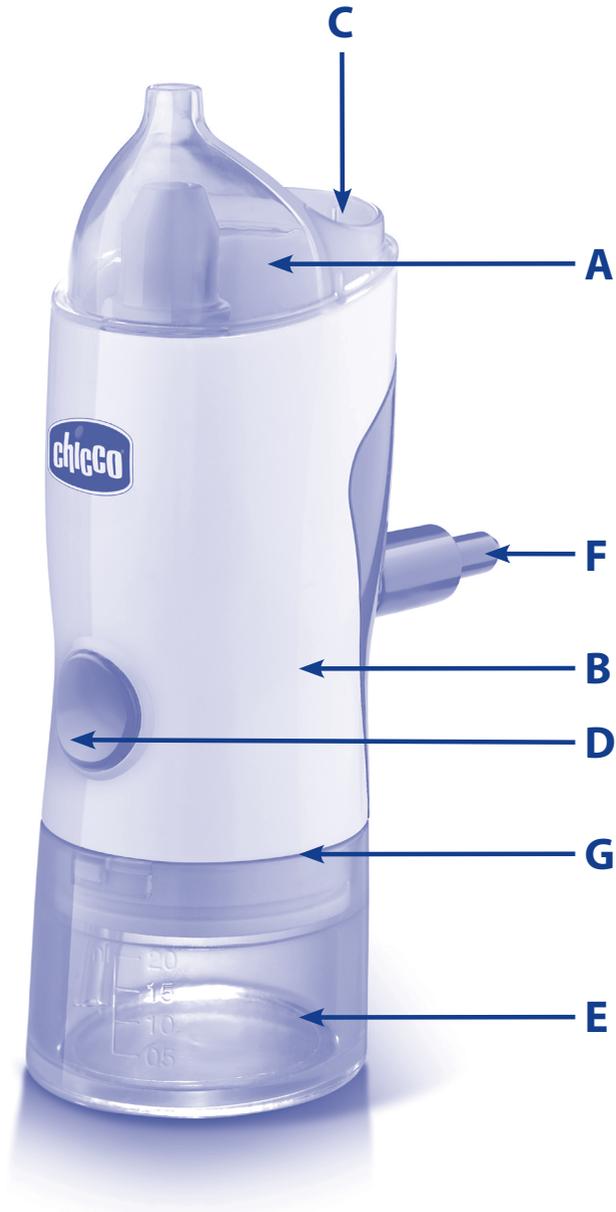
Fig. 1

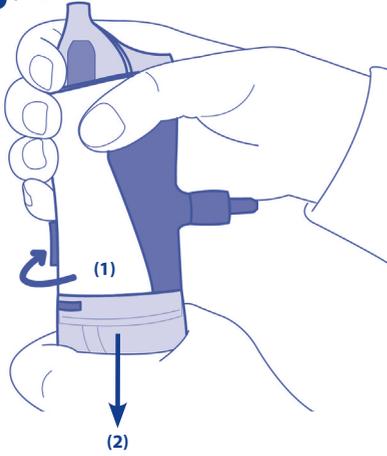
Fig. 2

Fig. 3

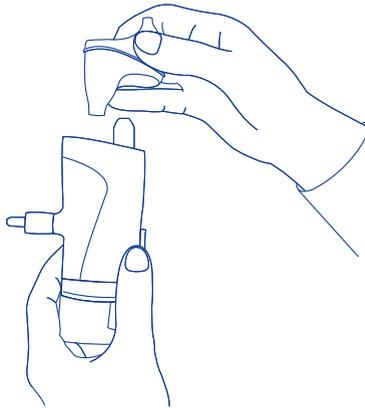
Fig. 3B

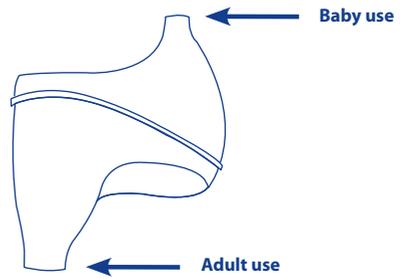
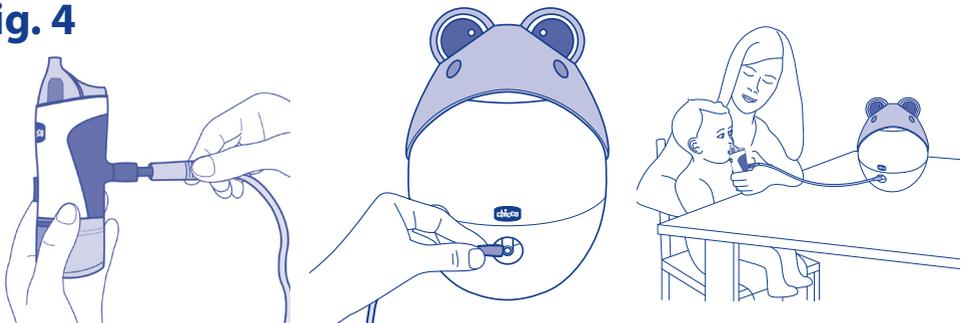
Fig. 4

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 Numero verde consumatori Italia 800 188 898

EN INSTRUCTIONS MICRONISED NASAL IRRIGATOR

WARNING! Carefully read the information in this manual before using and keep for future reference.
WARNING! The product packaging contains important information. Keep safe for future reference.

INTRODUCTION

The micronised nasal irrigation device is ideal for washing the nasal cavities and nasopharynx. The device has been specially designed to make treatment more practical and more effective, minimising medicinal waste. In addition, thanks to the practical dual-use tip, the irrigation device can be used by children and adults. One instrument for the whole family.

DEVICE COMPONENTS

- A - Dual-use tip (adults/children)
- B - Ergonomic body
- C - Drop collector
- D - Pump button
- E - Saline solution tank (capacity MAX 20ml - MIN 5ml)
- F - Nebulizer compressor air tube connector
- G - Seal

WARNINGS

Only use with pneumatic nebulizer devices (with compressor) from the Chicco line. For the correct use of nebulizer devices consult the instructions contained within the packaging.

⚠ WARNING: for the correct and safe use of the device never insert the tip in the nostril.

- Never fill the tank or use the nasal irrigation device with substances or liquids different from regular saline solution as these could cause irreparable damage to the device
- Only use the device for the purposes for which it was designed: washing the nasal cavities and nasopharynx. Using it in other ways could be dangerous.
- Do not use the device if it is damaged or worn in any way.
- If used by children or persons with reduced physical or mental capacities, it is essential that they are properly supervised or informed as to the correct use of the device by a person responsible for their safety.
- Do not use the device without the tip (A).
- If the device is sold by the original owner or given to third party users, the latter must be given the instructions manual illustrating how to safely use the device together with the safety warnings contained in it, which must always accompany the device.
- If you have any doubts about the content of this instructions manual, please contact your retailer or Chicco customer services.

⚠ WARNING: the nasal irrigation device and its components must not be placed in boiling water as this could cause irreparable damage.

SYMBOLS KEY

 Symbol of conformity with MDD 93/42/EEC directive + 2007/47/EC

"MANUFACTURER"

Lot Number

Catalogue Number

Storage and transport temperature symbol

Keep dry

WARNING!

Do not dispose of this device or its components as normal urban refuse but separate the waste as requested by the laws in force in each country.

Warning: read the instructions

Device date of production: the second pair of digits in the lot number identifies the year, the third pair identifies the month.
(**LOT** 1016100001; 16 = 2016; 10 = October)

PREPARING AND USING THE DEVICE

- Remove the transparent tank by turning clockwise until completely unscrewed and then pulling downwards (fig. 1);
- Insert MAX 20ml of saline solution in the tank (E), (fig. 2). If, according to a medical prescription, it is necessary to use a greater quantity of saline solution, divide this quantity into multiple sittings of MAX 20ml.
- Fully insert the two rectangles in relief at the bottom of the device body (B) in the grooves of the tank (E), turn the tank (E) anti-clockwise until it is screwed in tightly.
- Insert the dual use tip (A) in the body of the device, choosing the adult or child version (fig. 3);
- Connect one end of the air tube (not included with the product) to the compressor air outlet and the other to the air connector of the nasal irrigation device (fig. 4);
- Plug the power cable of your Chicco piston aerosol device into a mains socket;
- Check that all parts are correctly inserted and connected together;
- Turn on the nebulizer device by moving the switch to the on position (see device instructions);

⚠WARNING:

When the nebulizer is on:

- a) a small puff of air is continuously emitted from the ventilation hole under the air tube connector (F). This is necessary for the correct functioning of the nasal irrigation device and is completely normal.
- b) a small puff of air and drops of saline solution may come out from the nebuliser nozzle hole even if the pump button is not being pressed. This is normal.

- Bring the dual use tip (A) up to the nostrils, press and hold the pump button, and carry out the treatment session. During treatment, we recommend that you close your other nostril using your finger.
- If the pump button is released, the irrigation is suspended. To begin again, press the button again and hold down for a continuous flow.
- The quantity of saline solution inserted in the tank must be evenly distributed between the two nostrils. Breathe normally through the nose during treatment. If necessary, momentarily interrupt irrigation to blow your nose.
- During the session, a certain amount of saline solution and mucous is collected in the collector (C), which must be disposed of at the end of the sitting. This is normal and has no bearing on the results of the treatment.
- When the session has finished, turn off the nebulizer device and carry out the cleaning/disinfection operations described in the relevant paragraph.

CLEANING AND DISINFECTING

Before each use:

- The nasal irrigation device must be emptied and cleaned straight after every use to avoid the build up of incrustations that could compromise the correct functioning of the accessory.
- Disassemble the device into its removable parts and thoroughly wash both the inside and outside of the device body, the tip and the tank with warm water; if necessary disinfect with denatured alcohol and rinse carefully.
- Make sure that all components are completely dry before storing them in a clean, dry place away from sources of light and heat.

⚠WARNING: the nasal irrigation device and its components must not be placed in boiling water as this could cause irreparable damage.

TECHNICAL DATA

With nebulizer device flow	16	lpm
Irrigation device flow (when on)	8	lpm
Irrigation device pressure (when on)	0.36	bar

WARRANTY

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions for use. The warranty shall not therefore apply in the case of damages caused by improper use, wear or accidental events. For the duration of warranty on conformity defects please refer to the specific provisions of national laws applicable in the country of purchase, where provided.

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Artsana S.p.A – via Saldarini Catelli, 1