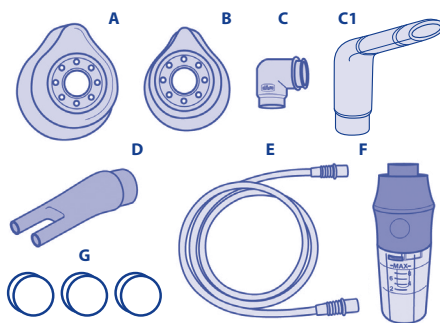
## Kit accessori per aerosol a pistone Super Soft

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## INSTRUCTIONS

**WARNING:** Before using the kit please read these instructions carefully and keep them for future reference.

### ACCESSORIES

- A - Adult mask
- B - Children's mask
- C - Mask connector
- C1 - Mouthpiece
- D - Nose fork
- E - Connection tube
- F - Nebuliser cup
- G - Filter

### PREPARING THE NEBULISER CUP:

- Turn the top part of the nebuliser cup anti-clockwise. Insert the drug in the bottom of the tank (fig.1).
- **WARNING:** do not insert a quantity of drug/saline solution that is below or above the minimum or maximum level indicated on the bottom of the nebuliser cup tank.
- Close the nebuliser cup by turning the top part of the nebuliser cup in a clockwise direction. Attach the air tube to the nebuliser cup connector and the compressor air outlet (fig.2). Attach the desired accessory (mask, mouthpiece etc.) (fig.3), switch on the appliance and start aerosol therapy. When the aerosol jet becomes intermittent, interrupt the treatment for a few seconds to

allow the suspended drops of solution to fall into the bottom of the nebuliser cup tank. Continue with the treatment session and finish when the flow becomes intermittent again.

### CLEANING THE NEBULISER CUP AND ACCESSORIES

Detach all the nebuliser cup components and dip them in boiling water for about 5 minutes. Do not boil the masks. Do not boil the masks. Rinse all accessories under running water. Cold disinfect all parts/accessories that come in contact with the patient using denatured alcohol. Make sure that all components are completely dry before storing them in a clean, dry place away from light and heat.

### ENVIRONMENTAL CONDITIONS DURING USE:

Relative humidity: 15% ÷ 85% RH  
Temperature: 5°C ÷ 40°C  
Atmospheric pressure from 700hPa to 1060hPa

### CONDITIONS FOR STORAGE AND TRANSPORTATION:

Relative humidity: 15% ÷ 85% RH  
Temperature: -25°C ÷ 70°C  
Store in a cool, dry place.

**DEVICE DATE OF PRODUCTION:** the second pair of digits in the lot number identifies the year, the third pair identifies the month. (LOT 1016100001; 16 = 2016; 10 = October)

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