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

ELECTRIC BREAST PUMP


A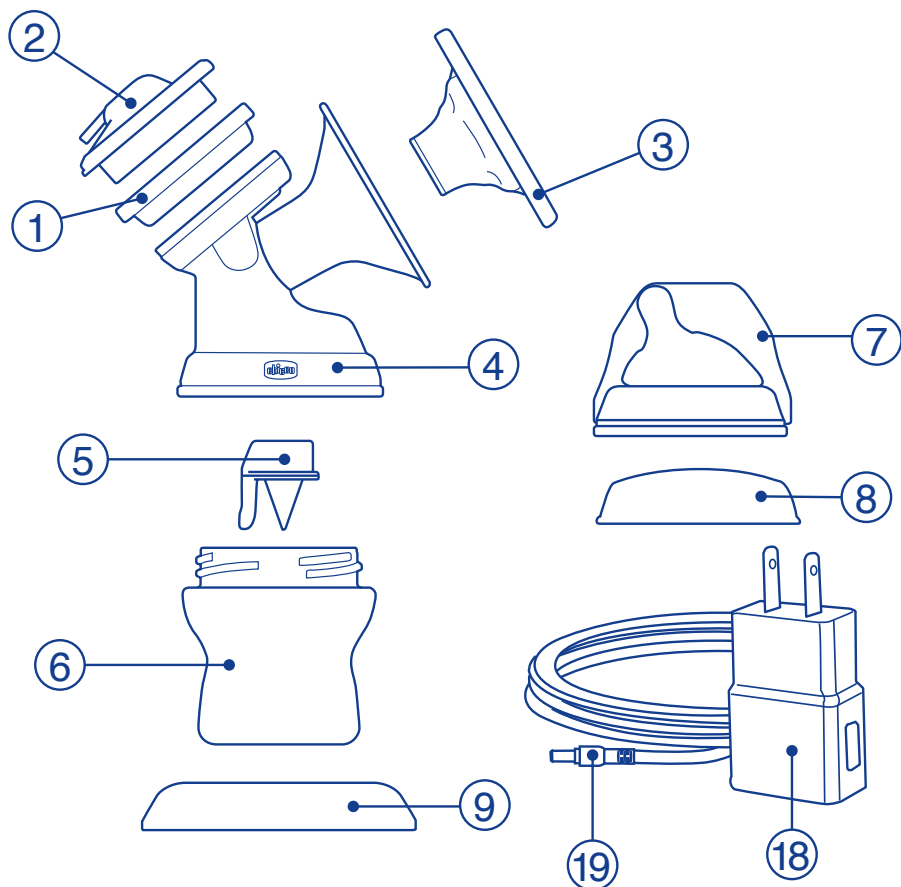

B

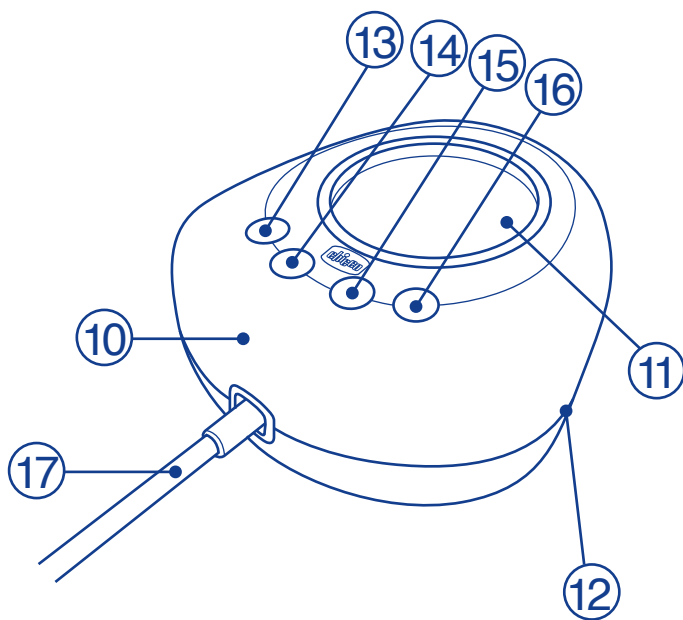C

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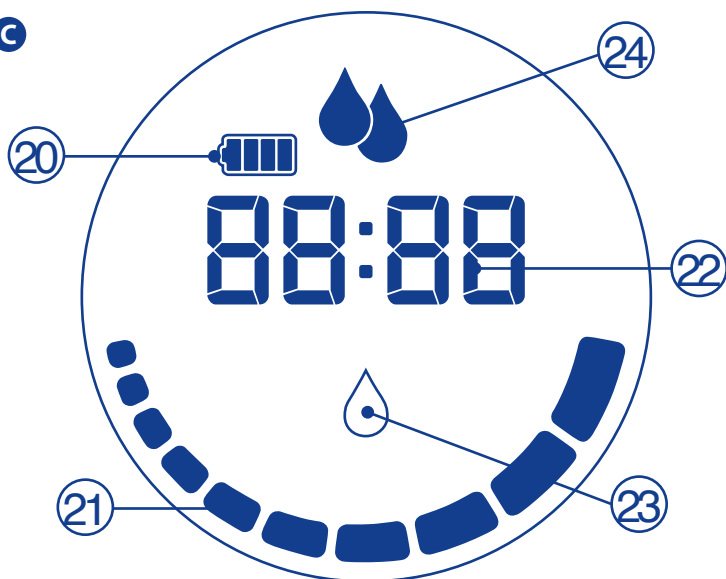EN INSTRUCTIONS FOR USE

NaturallyMe Electric Breast Pump

Please read these instructions carefully before use and keep them for future reference.

SAFETY WARNING. WARNING!

- This product is intended for domestic use only, and must be used in accordance with the instructions for use provided. Any other use is to be deemed improper and therefore dangerous. Artsana declines all responsibility for any damage caused by inappropriate, incorrect or unreasonable use of the appliance.
- The appliance is for personal use and must not be sold by the purchaser or lent to other users.
- This appliance and its components must not be exposed to direct sunlight, rain and other elements. The mains adaptor is for indoor use only and should therefore not be exposed to sunlight, rain or other elements.
- If the appliance is used outdoors, it must be powered only by internal batteries and must not in any case be exposed to sunlight, rain or other elements: failure to comply with this warning may expose the user to risk of electric shock.
- This appliance is not a toy. This appliance should not be used by children or persons with reduced mental faculties, or who do not have the required experience or knowledge. The product can be used by persons with reduced physical or sensory capacity as long as under supervision or after they have received instructions for the safe use of the appliance, and have understood the hazards involved.
- Keep the appliance, its components and the mains adaptor with the power cord connecting to the motor unit out of children's reach. The power cord connecting the mains adaptor and the motor unit, and the tube connecting the motor unit to the extraction kit, if left within children's reach, may cause strangulation. The appliance also contains small parts which, if ingested by a child, may cause suffocation.
- Please ensure that children do not play with the appliance, its components or the mains adaptor.
- Before connecting the mains adaptor to the mains socket, ensure that the information on the power data plate and the mains adaptor are compatible with your electricity distribution network and the corresponding socket. The reference power data plate can be found on the mains adaptor.
- If the mains adaptor plug supplied with the appliance is not compatible with the mains socket, use only certified adaptors, or contact our qualified technical personnel.
- Connect the mains adaptor to a socket that is easily accessible only by an adult.
- When disconnecting the appliance from the mains electricity supply, first pull the mains adaptor plug out of the socket and only then disconnect the mains adaptor outlet cable from the motor unit. Do not pull on the cord of the mains adaptor or the appliance itself if connected, to disconnect the mains adaptor

from the socket.

- Do not perform any cleaning or sanitising operations on the breast pump while it is operating, when it is powered by alkaline batteries (not supplied) or when it is connected to the mains through its mains adaptor. Cleaning and sanitising operations must be carried out in accordance with these instructions for use.
- Before each use, check that the membrane is not damaged: it must never be torn or cut. If it is torn or cut, do not use the appliance.
- Any cleaning and sanitising to be performed by the user must not be carried out by children and/or people with reduced mental faculties or lacking experience or the required knowledge.
- Do not wet the appliance or its mains adaptor or put it into water or other liquids and never place it under running water. If any liquid has entered the appliance or the mains adaptor, contact our qualified technical personnel before further use.
- If the appliance, the mains adaptor or its plug show any sign of damage, they must not be used. If the appliance or the mains adaptor show signs of damage, contact our qualified technical personnel before further use.
- The cable connecting the mains adaptor to the motor unit cannot be replaced and therefore, if damaged, the mains adaptor must no longer be used.
- Warning, never use a mains adaptor of a type different from that supplied by Artsana S.p.A. with the appliance (model LXCP12A-006100DEH, manufactured by: Shenzhen Longxc Power Supply Co. , LTD), since doing so would make the appliance non-compliant with the applicable specific technical standards, impairing its safety and integrity. Contact our qualified personnel in the event of failure of the mains adaptor.
- Do not attempt to open or repair the appliance or the mains adaptor when faulty or damaged. Any repair/service carried out directly by the user may lead to severe risks to her or to any third party's safety as well as irreparably damaging the appliance and entails immediate termination of the warranty.
- If the packaging appears to be damaged, check the state of all the components and parts of the appliance. If the mains adaptor and the motor unit proved to be undamaged, wash all the components of the kit following the instructions given in the paragraph "CLEANING" as a precaution.
- Before use, remove and dispose of all plastic bags and packaging materials and keep them out of reach of children; they may be a source of danger.
- Make sure the unit is placed on a level, stable surface, away from cradles and cots, when used in order to prevent spills or falls.
- Do not place the appliance on delicate surfaces or surfaces that may be sensitive to humidity and heat.
- Do not use the appliance while taking a bath or shower, or when in a damp place, or close to bath-tubs, sinks, washbasins, or in any other situation in which there are liquids that may come into contact with the appliance.
- Make sure that, when in use, the kit is not too from the engine body so that the latter does not fall.
- When using any electrical appliance, it is necessary to follow a few basic safety rules:
 - Never touch the appliance with wet or damp hands.
 - Keep the appliance and the mains adaptor away from heat sources.

- Do not touch the appliance and the mains adaptor: if they accidentally fall into water, or accidentally come into contact with water or milk. Immediately disconnect the power supply using the master switch in your home and then disconnect the plug of the mains adaptor from the socket before touching the appliance. Do not use the appliance but contact qualified personnel.
- If the appliance and/or the mains adaptor and/or the connection cord between the mains adaptor and the motor unit are faulty and/or not operating correctly, immediately switch off the appliance, disconnect the plug of the mains adaptor from the mains socket, do not attempt to tamper with or use the appliance but contact qualified personnel.
- If the appliance is not going to be used for a prolonged period, it should be thoroughly cleaned and the batteries (if installed) removed before storing it in a clean and dry place away from light and heat sources and according to the storage and transport instructions.
- If you decide not to use the appliance any more, remove the alkaline batteries, if present, and remove the output cable from the mains adaptor (after disconnecting the plug from the mains socket) to make it unusable, dispose of it as required or by the laws in force in the countries where it is sold.
- Always remove alkaline batteries if you decide not to use the device for a long time. Failure to comply with this warning may cause leaking of acid with consequent damage and chemical contamination of the appliance.
- To avoid any risk of fire or electric shock, ensure that the cable connecting the mains adaptor and the appliance does not pass under any carpets or close to radiators, stoves and heaters.
- Operate the appliance at a temperature range between +5°C and +40°C. Use outside these temperature limits, and in particular above the upper limit, may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
- The appliance has no protection against oxygen-rich environments: we therefore recommend that the appliance should not be used in environments, including domestic areas, where there is a patient using home oxygen therapy.
- The appliance must not be used continuously for more than 30 minutes. When you have finished using it, the appliance must be switched off for at least 30 minutes. If the appliance is used for longer than the permitted maximum operating time, even though it has a system to automatically switch it off after 40 minutes, this may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.

WARNINGS - REMOVING AND INSERTING BATTERIES

- Non-rechargeable batteries must only be replaced by an adult.
- Only use alkaline non-rechargeable batteries of the same or equivalent type recommended for the correct operation of this device.
- Never mix different types of batteries or new and used batteries.
- Do not use rechargeable batteries for this appliance, as they may reduce or prevent its correct operation.
- Do not use lithium or rechargeable batteries. This appliance is not designed to operate with these types of battery.

- Using lithium or rechargeable batteries may create hazardous conditions.
- Always remove run-down non-rechargeable batteries from the device to prevent leaks of liquid that may damage it and/or anything near to it. Leakage of liquid may cause chemical contamination of parts designed to come into contact with food.
- If the batteries are leaking liquid, replace them immediately, making sure you clean the battery compartment and thoroughly wash your hands if they have come into contact with the liquid. Check that any leaked liquid has not come into contact in any way with the parts of the kit, in particular with those designed to come into contact with the milk.
- Do not throw run-down batteries into a fire or release them into the environment or dispose of them as normal urban waste. Batteries must be disposed of in compliance with the laws in force in the country where the appliance has been sold.
- Remove the batteries from the appliance before you dispose of it.

SYMBOLS KEY

Production Batch

Product catalogue reference code

Instructions for use

WARNING!

Read the Instructions for use – Follow the Instructions for use

CLASS II Appliance

For indoor use only

ON/OFF (press-press)

Alternate current

Direct current

Mains adapter polarity connection

Battery insertion polarity

INTENDED USE AND INDICATIONS FOR THE ELECTRIC BREAST PUMP

The Chicco NaturallyMe electric breast pump is intended for extracting breast milk.

THE BREAST PUMP AND BREASTFEEDING

Breastfeeding is a very important stage in a baby's and in a mother's life. For babies, latching on and sucking means not only satisfying a primary physiological need, but also, and more importantly, a precious moment in their relationship with their mothers, full of emotions and tactile, auditory, taste, and olfactory sensations. Satisfying this need gives a great feeling of tranquillity and well-being to both the mother and her child. Breast milk contains all the nutrients required to meet infants' nutritional and psychological needs at least for the first 6 months of their lives.

Chicco's electric breast pump helps extend breastfeeding, in the event of excess milk or when you cannot always be with your baby. In such cases you can extract milk, store it, and give it to baby when required.

To guarantee the greatest flexibility and convenience, the electric breast pump operates both with AA size 1.5V alkaline batteries (not supplied) and with a special mains adaptor (model LXCP12A - 006100DEH, manufactured by: Shenzhen Longxc Power Supply Co., LTD).

- Product features might be subjected to change without notice.
- If you have any doubts on the contents of this instruction manual, please contact your retailer.

LIST OF COMPONENTS

- 1- Membrane
- 2- Membrane cover
- 3- Soft silicone cover
- 4- Pump body
- 5- Valve
- 6- NF feeding bottle
- 7- Om teat with cap and ring
- 8- Airtight cap
- 9- Base
- 10- Engine body
- 11- Display
- 12- Battery compartment
- 13- On/off button
- 14- Button for increasing level of extraction/stimulation
- 15- Button for decreasing level of extraction/stimulation
- 16- Stimulation/extraction button
- 17- Silicone tube
- 18- Power supply
- 19- Appliance power cable connector
- 20- Battery level indicator
- 21- -Extraction/stimulation level indicator
- 22- Timer
- 23- Stimulation indicator
- 24- Extraction indicator

PREPARATION

- Before using the Breast pump for the first time, please clean all the components* thoroughly as shown in the "**CLEANING**" paragraph.
- Then assemble the breast pump as follows:
- Push the valve (5) down into the circular duct in the base

of the pump body (4). (Fig. 1)

NB: fit the valve preferably with the tab positioned inside the pump body.

- Push the outer edge to prevent damaging the valve and make sure that both closing edges of the assembled valve are joined together. If not, reposition the valve, making sure that the two edges are correctly aligned.
- For greater comfort, you can fit the silicone cover (3) onto the pump body (4). Insert the silicone cover well into the cone of the pump body, and seal it around the outer rim of the pump body (Fig.2). Press the pads of the cover to expel any air trapped inside.
- Fit the membrane cover (2) onto the silicone membrane (1) and insert the membrane and cover (pre-assembled) onto the pump body (4) (Fig.3)
- Screw the NaturalFeeling feeding bottle (6) onto the threaded base of the pump body (4).(Fig.4)
- Use the practical base to lay the assembled product on a flat surface (9) and improve its stability.
- To use the breast pump with its mains adaptor (18), attach the connector (19) to the socket on the motor unit (10) and then insert the plug of the mains adapter to a wall socket. (Fig.5)
- To use the breast pump with alkaline batteries, open the cover of the battery compartment (12) and insert two AA size 1.5V batteries (not included) making sure you match the polarity markings. Close the compartment. The duration of the alkaline batteries may change based on the used brand and their charge status.
- **WARNING:** as a precaution, always remove the batteries from the engine body (10) if the breast pump is powered by the mains adaptor (18) and/or if it is not used for a long time.
- Connect the silicone tube (17) to the membrane cover (2) (Fig.6)
- Insert the rigid connector on the silicone tube (17) into the slot provided on the motor body (10) (Fig.7)
- Take care that you do not lose the connector, if it is lost you may not be able to use the appliance.

*Excluding the motor body (10) and mains adaptor (18) and the silicone tube (17).

USING THE BREAST PUMP

- Before each use, sanitise all the components* with a cold sanitizing method using sterilising liquids (following the manufacturer's instructions), or a hot one (steam) using specific devices, or by boiling them (5 minutes), and letting all the hot sanitized components fully cool down before use.
- Wash your hands and assemble the breast pump as shown in the **PREPARATION** paragraph.
- The appliance is designed to operate only with the components supplied in the original packaging and with a 150-ml feeding bottle. Use with other components or with feeding bottles with higher capacities may not guarantee the stated performance.
- Cleanse your breast appropriately using water or one of the handy Chicco breast wipes.
- In order to facilitate the milk flow, you can take a hot shower or apply a warm cloth on your breast just before expressing milk or breastfeeding your child.

- Sit down comfortably and relax.
- Fit the cup over your breast ensuring that the nipple is centred properly. You can use the breast pump both with and without the soft silicone cover (3); We recommend trying both configurations before choosing the best and most comfortable one for you.
- Press the on/off button (13) on the motor body (10) to switch on the breast pump, the display (11) will light up.
- If you are using the breast pump powered by alkaline batteries, the indicator on the display will show the battery charge status (20). If it is powered by the mains adaptor, this indicator will not appear. The battery indicator shows 4 levels based on the battery charge status.

NB: while using the breast pump, to ensure correct operation of the appliance, take care not to cover the hole on the left-hand side of the pump body (4) with your hand.

NB: when switched on, the breast pump always starts in stimulation mode to facilitate milk extraction, using the factory settings (level 5). The stimulation indicator (23) will appear on the display.

- After approximately 2 minutes of stimulation, the appliance automatically switches to the extraction mode using the factory settings (level 5). The extraction indicator (24) will appear on the display.
- You can stop the stimulation stage and switch to the expression stage by briefly pressing the on/off button.
- If necessary, repeat stimulation by pressing the stimulation/extraction button (16) again. Press the stimulation/extraction button (16) to return to extraction.
- You can use the "+" (15) and "-" (14) buttons on the keypad to adjust extraction/stimulation based on the level of pull perceived on your breast and/or based on the amount of milk expressed. We recommend adjusting the extraction/stimulation rate gradually until you reach the level best suited to you.
- If the level of extraction/stimulation chosen causes discomfort or pain, suitable reduce it by pressing the "-" button (14).
- You can alternate milk extraction from both breasts.
- Press the on/off button (13) to switch the breast pump off.
- To avoid all danger deriving from spilt liquid (milk), we recommend that you never fill the feeding bottle beyond the highest level indicated on its graduated scale.
- **WARNING:** immediately after each single use, carefully clean all the components* as shown in the **'CLEANING'** paragraph.
- To feed the milk immediately to your baby, unscrew the feeding bottle from the pump body (4) to remove it. Screw the ring with the teat (7) (included in the pack) onto the feeding bottle. This way, you will obtain a handy, ready-to-use baby feeding bottle.
- If, on the contrary, you close the bottle with the airtight cap (8) (included in the package) you can store the expressed milk in the fridge for up to 48 hours at 4°C or for up to 3 months at -18°C in the freezer.
- In order to avoid the proliferation of bacteria, it is advisable to feed the baby immediately after the milk has been collected, or to immediately store the collected milk in the most appropriate way.
- **WARNING:** When the breast pump is not used, always disconnect the mains adaptor from the mains.

*Excluding the motor body (10) and mains adaptor (18) and

the silicone tube (17).

CLEANING

- Before and after each single use clean and sanitize the silicone cushion (3), the pump body (4) and the valve (5). Clean and sanitize the bottle (6) and its accessories (7)-(8) as illustrated in the related instructions.
- Before carrying out any cleaning operation, disconnect, if connected, the mains adaptor from the mains and the power cord connector from the socket (19) on the motor body.

- Separate all the components from the breast pump.

NB: remove the valve (5), located on the circular duct inside the base of the pump body (4) by gently pulling the tab on the side of the valve.

- **WARNING:** always remove the valve gently and do not use any tool to clean it since this may damage it, making the breast pump impossible to use.

NB: to remove the silicone tube, first detach the membrane (1) and the membrane cover from the pump body, to avoid any spillage. Then, holding the end inserted into the membrane cover, gently pull it off.

- Carefully wash all the components* using hot water and washing up liquid, thoroughly rinse and dry carefully using a soft and clean cloth.
- Do not use abrasive products to clean the breast pump and its components.
- Clean the motor body and the mains adaptor with a soft, clean and dry cloth. Warning! Do not wet the appliance or put it into water or other liquids.
- The tube may be rinsed and suitably dried.
- All the components, with the exception of the motor body (10), the mains adaptor (18), the valve (5), the membrane (1), the silicone cover (3) and the tube (17) may also be washed in a dishwasher using a delicate programme. This washing method, which is more aggressive than hand washing, may discolour or dull and degrade the material over time.
- Also clean and sanitise the feeding bottle (6) and its accessories (7)-(8), according to the instructions included with the bottle.
- After washing, always store the product in a clean, dry place.
- *Excluding the motor body (10), mains adaptor (18) and the silicone tube (17).

ACCESSORIES:

- NaturalFeeling 150 ML feeding bottle [REF] 00080711000xxx / [REF] 00080811000xxx.
- NaturalFeeling Milk Container [REF] 00002257200000
- Where XXX=variable identification code identifying the country of sale.
- Two nursing pad to be used only to absorb excess breast milk are supplied free with the appliance (REF 20061770000).
- No accessories other than those recommended must be used.

Problem	Cause	Solution
The breast pump does not pump or pumps very little.	Valve not assembled or not properly assembled.	Assemble the valve correctly.
	Cup and motor body not assembly correctly.	Check that you have performed the procedure described in the "PREPARATION" paragraph correctly. Make sure that the silicone tube is firmly connected to both the engine body and the membrane cover. Check that the tube is not damaged. Do not twist the tube during milk expression.
	The cup does not fit well on the breast	Fit the pump body over your breast, taking care that your nipple is properly centred. Try with/without the soft silicone cover.
	Suction level not correctly adjusted.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "+" or "-" buttons.
	Run-down and/or insufficiently charged batteries.	Replace the batteries.
Milk expression is painful.	Suction level too high.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "-" button.
The breast pump does not switch on	Failure of mains adapter and/or appliance.	Check that mains power is present. Check the position of the batteries in the battery compartment.
	Incorrect connections.	Check all the connections.
	Run-down batteries.	Replace the batteries.

TECHNICAL SPECIFICATIONS

- ELECTRIC BREAST PUMP Mod. 06836M **REF** 00009199000590
- Maximum vacuum level in extraction mode: -0.35 ± 0.03 bar**
- (** Values measured with sample cylinder. CI 95%)
- Type of appliance: intermittent operation; 30 minutes ON - 30 minutes OFF
- Motor body weight: 165 g excluding batteries
- Engine body's dimensions: Base 98 mm X Height 105 mm X Depth 70mm
- Made in China

Storage, preservation and transportation conditions:

- Ambient temperature: $-20 / +60^{\circ}\text{C}$
- Relative humidity: 10 to 85%

Operating conditions:

- Operating temperature: $+5^{\circ}\text{C} / +40^{\circ}\text{C}$
- Relative humidity: 30 to 85%

CURRENT ADAPTER (NETWORK ADAPTER):

100V - 240V ~ 50/60 Hz 0.5A Output: 6V 1A

ELECTRIC BREAST PUMP Input 6V 1A

Works with the power adapter or 4 x AA 1.5 V alkaline batteries each (not included)

WARRANTY

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions for use. The warranty shall not therefore apply in the case of damages caused by improper use, wear or accidental events. For the duration of warranty on conformity defects please refer to the specific provisions of national laws.

Manufacturer: Artsana S.p.A - via Saldarini Catelli, 1 - 22070 Grandate (Como) - Italy

Made in China

EN INSTRUCTIONS FOR USE

NaturalFeeling feeding bottles

Please read these instructions carefully before use and keep them for future reference.

For your child's safety and health.

WARNING:

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay. This may occur if the baby is allowed to use the feeding bottle for long periods of time, even when drinking unsweetened drinks.
- Always check food temperature before feeding. This is to prevent the risk of scalding.
- Always inspect the teat before each use by pulling it in every direction. Throw away at the first sign of damage or weakness.
- In any case, we recommend replacing teats at least every two months.
- Do not leave teats in direct contact with sunlight or a heat source; do not leave it in disinfectant (sterilisation solution) for longer than recommended as this may damage them.
- Check that the flow is suited to the baby's requirements, paying attention to any sucking problems or to milk coming out of the baby's mouth.
- Inspect the feeding bottle before each use, making sure there are no broken parts or sharp edges.
- Give your baby food and drink according to your paediatrician's recommendations and the preparation methods indicated by the makers.
- Do not allow your baby to run or play with the product.
- Use only genuine components and spare parts from the NaturalFeeling range.
- Keep all components not in use out of the reach of children.
- Remember that breastfeeding is best for your child.

INSTRUCTIONS – Cleaning

- Wash before first use.
- Before using for the first time, immerse in boiling water for 5 mins. This ensures that it is sterile.
- Wash with hot water and washing up liquid before every use, then rinse thoroughly.
- It is preferable to wash the bottle by hand, avoiding abrasive products; washing it in the dishwasher may cause the product to wear more quickly.
- If bottle brushes are used, make sure you do not damage the more delicate parts of the product (e.g. teat openings and valves).
- Sanitise before use.
- Make sure that no liquid remains inside after washing or sanitising.
- Store in a clean, dry place.
- Pigmentation in some foods (e.g. tomato sauce) and strongly-coloured drinks (e.g. tea, fruit juices) may alter the colour of the product.

SANITISING








- During boiling or hot sanitising, the feeding bottle must be opened and disassembled in order to prevent it from de-forming.
- The product may be sanitised using cold sterilisation solutions, or hot methods (steam) using suitable equipment or by boiling for 5 minutes, avoiding contact with the metal walls of the container used.
- UV sterilisers are not recommended.

Assembly

- Make sure that the product is assembled correctly before use to ensure the correct functioning of the feeding bottle and to prevent any leaks.

Warming the feeding bottle

- To warm the contents, we recommend using the special feeding bottle warmers after removing the cap.
- Take extra care when microwave heating:
- Insert the bottle only after removing the ring, and teat: Risk of explosion.
- If you use a combination microwave oven, ensure that the grill is turned off.
- Always stir heated food to ensure even heat distribution and test the temperature before serving to prevent scalding your baby.
- Do not heat the product in traditional ovens or on hot plates or open flames.
- It can be placed in a freezer. Remove the teat and use the cap provided for storage.

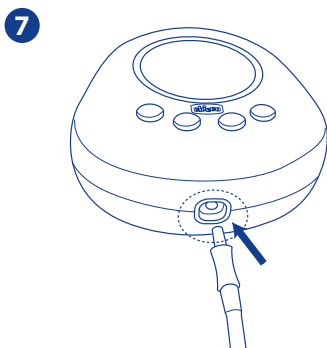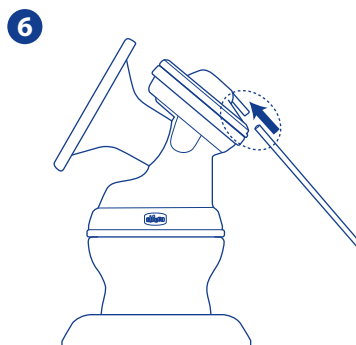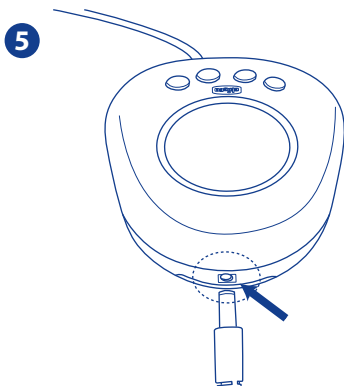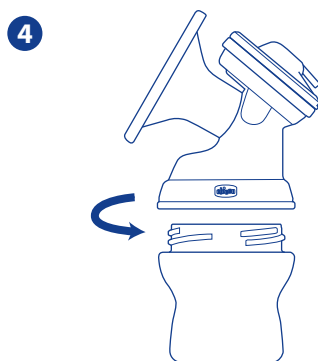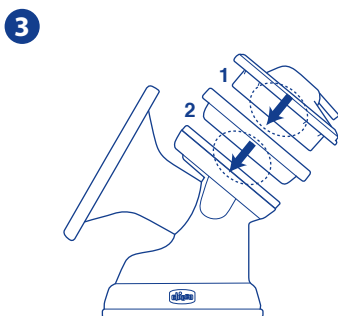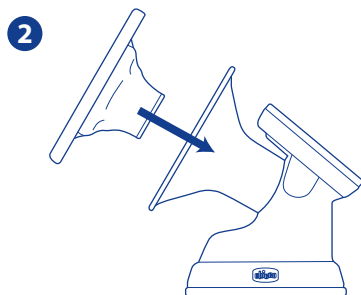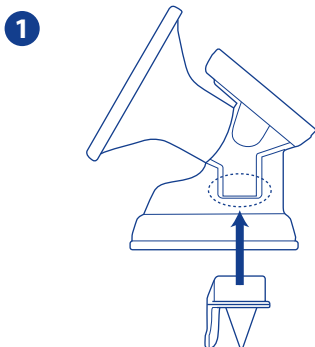NOTE

NOTE

[illegible]

46 009199 059 000_00_1744

REF 00009199000590

Rev. 1744

Made in China - Fabricado en China

Artsana S.p.A – via Saldarini Catelli, 1 - 22070 Grandate (Como) – Italy
www.chicco.com