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**WEARABLE ELECTRIC BREAST PUMP**

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**TRAGBARE ELEKTRISCHE MILCHPUMPE**

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# INSTRUCTIONS FOR USE

## WEARABLE ELECTRIC BREAST PUMP

Please read these instructions carefully before use and keep them for future reference. Intended use and indications for the electric breast pump.

## SAFETY WARNINGS

### WARNING!

- This appliance may be used by children of 8 years of age and older and by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge, if they have received supervision or instructions on how to use the appliance safely and understand the risks involved.
- Children must not play with the appliance.
- User cleaning and maintenance must not be carried out by children, unless they are over 8 years old and under supervision.
- Keep the appliance and the power supply cord out of the reach of children below 8 years of age. The USB cable could cause strangulation if left within reach of children.
- This product is intended for domestic use only by an adult, and must be used in accordance with the instructions for use provided. Any other use is to be deemed improper and therefore dangerous. Artsana declines all responsibility for any damage caused by inappropriate, incorrect or unreasonable use of the appliance.
- The appliance is for personal use and must not be sold by the purchaser or lent to other users.
- This appliance and its components must not be exposed to direct sunlight, rain and other elements.
- Please ensure that children do not play with the device or any components.
- Always place the appliance only on a flat, stable surface, away from cribs and cots.
- Before connecting the mains adaptor to the mains socket, ensure that the information on the power data plate and the mains adaptor are compatible with your electricity distribution network and the corresponding socket.
- Use only certified network adapters. AC adapters - DC 5.0V, 2000mA can be used to charge the device.
- Connect the charging cable to a mains adaptor and a socket that is easily accessible only by an adult.
- When disconnecting the device from the mains electricity supply, first pull the mains adaptor plug out of the socket, or the charging cable from the USB socket and only then disconnect the USB cable from the motor unit.

Do not pull on the USB cable or the device itself if connected, to disconnect the mains adaptor from the socket.

- Do not clean or sanitise the breast pump when it is in operation, when it is battery-operated or when connected to the mains. Cleaning and sanitising operations must be carried out in accordance with these instructions for use.
- In order to avoid the proliferation of bacteria, it is advisable to feed the baby immediately after the milk has been collected, or to immediately store the collected milk in the most appropriate way.
- Clean and sanitise the cup, milk container, valve and connector before each use.
- Before each use, check that the membrane is not damaged: it must never be torn or cut. If it is torn or cut, do not use the appliance.
- Any cleaning and sanitising to be performed by the user must not be carried out by children and/or people with reduced mental faculties or lacking experience or the required knowledge.
- Do not wet the device or put it into water or other liquids for any reason. If any liquid has entered the device, contact our free phone number (800-188-898) or qualified technical personnel before further use.
- If the device shows any sign of damage do not use it, or contact our free phone number (800-188-898) or qualified technical personnel before further use.
- Do not attempt to open or repair the device if faulty or damaged. In case of faults or damage, please contact our free phone number (800-188-898). Any repair/service carried out directly by the user may lead to severe risks to the safety of the user or any third party as well as irreparably damaging the device and entails immediate termination of the warranty.
- If the packaging appears to be damaged, check the state of all the components and parts of the appliance.
- Before use, remove and dispose of all plastic bags and packaging materials and keep them out of reach of children; they may be a source of danger.
- After removing any packaging materials, ensure that the device has not been damaged and follow the instructions provided under "CLEANING". If in any doubt, do not use the appliance and contact only technically qualified personnel, at the point of purchase or on our free phone number (800-188-898).
- Ensure that the device is placed on flat, stable surfaces to prevent it from tipping over or falling, once in use and before being emptied.
- Do not place the appliance on delicate surfaces or surfaces that may be sensitive to humidity and heat.
- Do not use the appliance while taking a bath or shower, or when in a damp place, or close to bath-tubs, sinks, washbasins, or in any other situation in which there are liquids that may come into contact with the appliance.
- When using any electrical appliance, it is necessary to follow a few basic

safety rules:

- Never touch the appliance with wet or damp hands.
- Keep the device and the power cable away from heat sources.
- Do not touch the device and power cable if they accidentally fall into water or come into contact with water or milk.
- In the event of a fault and/or malfunction of the device, switch it off immediately, unplug the adaptor from the mains socket, do not attempt to tamper with it, do not use it, and contact qualified personnel or the freephone number (800-188-898).
- The appliance must be cleaned and dried thoroughly at the end of every operating cycle.
- If the device will not be used for a prolonged time, it should be thoroughly cleaned before storing it in a clean and dry place away from light and heat sources and in compliance with the instructions on storage and transportation.
- To avoid any risk of fire or electric shock, ensure that the USB cable does not pass under any carpets or close to radiators, stoves and heaters.
- Operate the appliance at a temperature range between +5°C and +40°C. Use outside these temperature limits, and in particular above the upper limit, may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
- The appliance must not be used continuously for more than 30 minutes. When you have finished using it, the appliance must be switched off for at least 30 minutes. If the appliance is used for longer than the permitted maximum operating time, even though it has a system to automatically switch it off after 30 minutes, this may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
- Product features may be subjected to change without notice.
- If you have any doubts on interpretation of the contents of this instruction booklet, contact your dealer or call our free phone number (800-188-898).

#### LITHIUM BATTERY WARNINGS

- This product uses a lithium-ion battery as a power supply.

 If not in use for a long period, keep the unit in a cool, dry place and charge the battery every 3 months.

#### INDEX OF SYMBOLS

	Product catalogue reference code		Direct Current
	WARNING!		Complies with European Directive 2012/19/EU
	For indoor use only		Stand by (pressure)
	The appliance complies with the essential requirements of applicable EC Directives		Serial Number

## THE BREAST PUMP AND BREASTFEEDING

### THE BREAST PUMP AND BREASTFEEDING

Breastfeeding is a very important stage in a baby's and in a mother's life. For babies, latching on and sucking means not only satisfying a primary physiological need, but also, and more importantly, a precious moment in their relationship with their mothers, full of emotions and tactile, auditory, taste, and olfactory sensations. Satisfying this need gives a great feeling of tranquillity and well-being to both the mother and her child. Breast milk contains all the nutrients required to meet infants' nutritional and psychological needs at least for the first 6 months of their lives. The Chicco wearable electric breast pump helps extend breastfeeding, in the event of excess milk or when you cannot always be with your baby. In such cases you can extract milk, store it, and give it to baby when required.

### LIST OF COMPONENTS

1. Motor unit (2 pieces)
2. Milk container (2 pieces)
3. Membrane (2 pieces)
4. Connector (2 pieces)
5. Valve (2 pieces)
6. Cup (2 pieces)
7. Emptying hole
8. Nipple adaptors (21 mm and 24 mm) (2 pieces per type)
9. On/Off button
10. Level decrease button "-"
11. Level increase button "+"
12. Mode selection button
13. Battery level indicator
14. Stimulation mode indicator: 10 levels
15. Standard extraction mode indicator: 10 levels
16. Intense extraction mode indicator: 10 levels
17. USB port
18. Silicone cover
19. USB cable (2 pieces)
20. Fabric bag (2 pieces)

### SET-UP

- Before using the breast pump for the first time, thoroughly wash all components, with the exception of the motor unit (1) and the USB cable (18), as described under "CLEANING".

- Charge the motor unit (1) via the supplied USB cable (18) by plugging it into a power outlet using a compatible 5V 2A mains adaptor. Open the silicone cover (17) on the back of the motor unit, insert the USB cable (18) into the USB port (16), connect it to a compatible 5V 2A adaptor and then plug the latter into the mains socket.

- During recharging, the battery level indicator (13) flashes white and lights up solid white when charging is complete.

**WARNING:** The breast pump cannot be used while it is charging.

**WARNING:** Only charge the breast pump with compatible 5V 2A mains adaptors, otherwise the breast pump may be irreparably damaged.

**WARNING:** In case of prolonged non-use, recharge the battery at least once every 3 months.

- Then assemble the breast pump as follows:

- Fully insert the valve (5) onto the circular duct at the base of the connector (4) so that the cut of its edges is parallel to the cup (Step 1)

- Push the outer edge to prevent damaging the valve and make sure that both closing edges of the assembled valve are joined together. If not, reposition the valve, making sure that the two edges are correctly aligned.

- Assemble the silicone membrane (3) onto the connector (4) and make sure that the outer edge thoroughly overlaps with the edge of the connector housing (Step 2)

- Insert the connector inside the milk container by fastening it tightly to the membrane space at the top of the container. (Step 3)

- Close the milk container with the rigid cup by attaching it firmly to the connection. (Step 4). Make sure that the two parts are tightly

closed around the entire perimeter.

- Securely fasten the motor unit to the milk container and cup assembly (Step 5).
- On the back of the motor body there is an L marked on one motor and an R on the other to identify left and right respectively. The two motors are interchangeable, but these small references allow the user to identify which was used on one breast and which on the other, as the motor stores the most recent settings used before it was switched off.

### USING THE BREAST PUMP

- Before each use, sanitise all the components, except for the motor unit (1) and the USB cable (19), with a cold sanitizing method using sterilising liquids (following the manufacturer's instructions), or hot sanitizing (steam) using specific devices, or by boiling the components (5 minutes), and letting any hot sanitized components fully cool down before use.

**WARNING:** Do not sterilise inside microwave ovens as the components may be irreparably damaged.

- Before each use, it is advisable to check the battery charge status by pressing the on/off button once with the motor unit switched off and checking the indicator lights 13, 14, 15, 16:

- if all the indicators light up, the battery is fully charged,
- if indicators 13, 14 and 15 light up, the battery is almost fully charged,
- if indicators 13 and 14 light up, the battery is approximately 50% charged,
- if indicator 13 lights up, the battery is at minimum charge and recharging is recommended before using the breast pump to prevent it from dying during use,
- if indicator 13 flashes quickly, the battery is low and must necessarily be recharged before use as described under "PREPARATION".

- Wash your hands and assemble the breast pump as shown in the PREPARATION paragraph.

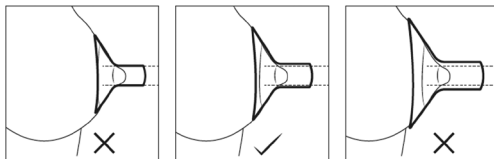
- Place the breast pump on the breast and make sure that no air passes between the breast and the cup so that the product works properly. Support the breast with your hand from below and hold the breast pump on the breast.

It is essential that the nipple is perfectly aligned with the hole on the cup (step 6).

- If the standard 27 mm cup is too large for your nipple, use a 24 mm or 21 mm silicone flange (7) from those included in the package, taking care to fit it completely around the cup after closing the milk container with the cup (Step 7).

- How can the user identify the most suitable flange?

The nipple must be able to move freely during use. If friction forms between the nipple and the cup/flange, then the latter is too small. If the areola is also sucked in, then it is too big.

- Once the breast pump is positioned on the breast, press and hold the on/off button (8) to switch the breast pump on.

- Indicators 13, 14, 15, 16 will light up for a few seconds to indicate the battery charge status as mentioned above.

Note: when switched on, the breast pump always starts in stimulation mode to facilitate milk extraction, using the factory settings (level 1). After displaying the battery charge status via indicators 13, 14, 15, 16 only the indicator corresponding to the stimulation mode (13) will light up.

- After approximately 2 minutes of stimulation, the appliance automatically switches to extraction mode at level 1 (factory settings). The indicator for standard extraction mode (14) lights up.

- You can stop the stimulation stage and switch to the standard ex-

pression stage by pressing the mode selection button (12) once. To switch to intense extraction mode (15), press the mode selection button (12) twice.

By repeatedly pressing the mode selection button (12), 3 different modes can be selected:

- 1- Stimulation (13)
- 2- Standard extraction (14)
- 3- Intense extraction (15)

- The intensity of stimulation and extraction can be adjusted by using the "+" (11) and "-" (10) buttons.
- If the level of extraction/stimulation chosen causes discomfort or pain, reduce it to a suitable level by pressing the "-" button (10).

Note: To avoid any danger deriving from spilt liquid (milk), never fill the milk container beyond the highest level indicated on the graduated scale (150 ml).

- The breast pump can be paused by pressing the on/off button (9) once, pressing a second time will resume the operation where it left off.
- To switch the breast pump off, press and hold the on/off button (9), otherwise the breast pump will automatically switch off after 30 minutes.

When the battery is low, indicator 13 will start flashing and the product will switch off shortly afterwards. The battery must necessarily be recharged before resuming use of the breast pump as indicated under "PREPARATION".

- After use, with the breast pump switched off, remove it from the breast while holding it upright.
- After placing the base of the container on a flat surface, remove the motor unit from the milk container (Step 8) while holding it upright so that the milk does not spill out.
- **WARNING:** the milk container is not suitable for milk storage.
- Empty the container by pouring the extracted milk through the emptying hole (7) directly into a feeding bottle/milk bag/container (Step 9).

## CLEANING

- Before carrying out any cleaning operations, check that the product is not connected to the mains and that it is switched off.
- Separate all the components from the breast pump. Note: remove the valve (5) by pulling it gently.
- **WARNING:** Make sure that the valve is always removed in a gentle way; do not use any tool for cleaning it because it may be damaged, making the breast pump impossible to use.
- Thoroughly wash all components except the motor unit (1) and the USB cable (19) with warm water and dishwashing liquid, rinse thoroughly and dry thoroughly with a soft, clean cloth.
- Do not use abrasive products to clean the breast pump and its components.
- Clean the motor unit with a soft, clean and dry cloth. Warning! Do not wet the appliance or put it into water or other liquids.
- The following can be washed in the dishwasher using a delicate wash cycle: milk container, connector, valve, membrane, cup and flanges. This more aggressive type of washing may cause the material to discolour or become dull and break down over time.
- Carefully clean the walls of the emptying hole with a bottle brush.
- After washing, always store the product in a clean, dry place.

If necessary, hand wash the fabric bag in lukewarm water following the instructions on the label that has the following washing symbols:

Hand wash

Do not bleach

Do not tumble dry

Dry flat

Do not iron

Do not dry clean

Fault	Cause	Solution
The breast pump does not pump or pumps very little.	Valve not assembled or not properly assembled.	Assemble the valve correctly.
	The cup does not fit well on the breast.	Fit the pump body over your breast, taking care that your nipple is properly centred. Try using the product by inserting the supplied flanges on the cup or, if already in use, try changing the flange or removing it altogether.
	Suction level not correctly adjusted.	Change the suction level until you achieve more comfortable stimulation and extraction by pressing the "+" or "-" buttons.
	Low or insufficiently charged battery.	Recharge with USB cable.
	Membrane not assembled or not properly assembled.	Assemble the membrane correctly in its housing
	Milk container and cup not properly closed and/or connector missing.	Check that the connector and all its accessories are present inside the milk container and that it is tightly closed with the cup.
	Suction level insufficient.	Change the suction level until you achieve more comfortable stimulation and extraction by pressing the "+" button.
Milk expression is painful.	Suction level too high.	Change the suction level until you achieve more comfortable stimulation and extraction by pressing the "-" button.
	The silicone cup does not fit well on the breast	Make the cup adhere well to the breast, checking that the nipple is well centred. Try using the product by inserting the supplied flanges on the cup or, if already in use, try changing the flange or removing it altogether.
The breast pump does not switch on.	Low or insufficiently charged battery.	Check that mains power is present once the product is plugged in.
	Incorrect connections.	Check all the connections.
	Low or insufficiently charged battery.	Recharge with USB cable.

#### TECHNICAL SPECIFICATIONS

- Maximum vacuum level in extraction mode:  $-0.35 \pm 0.03$  bar
- Type of appliance: intermittent operation; 30 minutes ON - 30 minutes OFF
- Single product weight: 206 g, including batteries
- Dimensions: 95 x 145 x 65 mm
- Made in China

#### Power supply:

rechargeable Li-ion battery 3.7V 1200mAh/power supply 5V DC 2A

#### DECLARATION OF EC COMPLIANCE

The device with the Chicco Breast Pump brand **REF 00009199700000 Mod. N0269** complies with the essential requirements and other relevant provisions of Directives 2014/30/EU (Electromagnetic Compatibility), 2014/35/EU (Low Voltage) and 2011/65/EU (RoHS). A copy of the EC declaration can be requested from the manufacturer: Artsana S.p.A via Saldarini Catelli, 1 - 22070 Grandate (Como) Italy.

#### **The Electric Breast Pump **REF 00009199700000 Mod. N0269** complies with Regulation (EU) 2023/1542**

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar appliance. The user is responsible for taking the appliance to a special waste disposal site at the end of its life. If the unwanted appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both the environment and health, and contributes towards the recycling of the product's materials. Illegal disposal of the product by the user will result in administrative sanctions provided by law. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

**The Electric Breast Pump REF 00009199700000 Mod. N0269 complies with Directive 2012/19/EU.**

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar appliance. The user is responsible for taking the appliance to a special waste disposal site at the end of its life. If the unwanted appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both the environment and health, and contributes towards the recycling of the product's materials. Illegal disposal of the product by the user will result in administrative sanctions provided by law. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

**WARRANTY**

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions for use. The warranty shall not therefore apply in the case of damage caused by improper use, wear or accidental events. For the duration of warranty on conformity defects please refer to the specific provisions of national laws

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Artsana S.p.A – via Saldarini Catelli, 1 - 22070 Grandate (Como) – Italy