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## CEROTTI POST PUNTURA AFTER BITE PATCHES

Eurostic S.p.A. - Viale Europa, n.2004/7 - 3° - Gennaia (Como) - Italy  
Distributed by Aransa s.p.a. Via Salomon Canevi 12/2001 - Caso, Milano (Italy)  
www.chicco.com

## CEROTTI POST PUNTURA AFTER BITE PATCHES

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- CALENDULA
- ZANTHOXYLUM
- LAVENDER
- VITAMIN E

**30 PCS**

VITIS VINIFERA (GRAPE) SEED OIL, LINSEED ACID, HELIANTHUS ANNUUS (SUNFLOWER) POLYMER, GLYCINE SOIA (SOYBEAN) OIL, MENTHOL, MINTA PERTITA (PEPPERMINT) OIL, CALENDULA OFFICINALIS FLOWER EXTRACT, ZANTHOLENE (ZANTHOLENE) OIL, LAVANDULA ANGUSTIFOLIA (LAVENDER) FLOWER EXTRACT, BENZOIC BENZYL ALCOHOL.

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**EN** These patches help alleviate the bothersome itching caused by external agents, such as insect bites, thanks to their soothing effect. In addition, they create a mechanical barrier that reduces friction, protecting the affected area from further irritation. Active ingredients Calendula and Zanthahene soothe and refresh the affected area. **WARNINGS:** product intended for adults and children over 3 years of age. Keep out of the reach of children under 3 years. In case of contact with the eyes rinse abundantly. For external use only. Store in a dry, cool place. Single-use only. Follow the instructions for use printed on the packaging.

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**EN** **Instructions for use:** 1. Clean and dry the skin in the affected area. Remove one plaster from the transparent support using the applicator. 2. Apply the plaster to the affected area. Avoid touching the adhesive part with your fingers. 3. Do not apply for more than 8 hours. Apply no more than once daily, using no more than 3 plasters at the same time. See illustrations on the side of the pack.

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