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Easy Breath Doccia Nasale

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## INSTRUCTIONS FOR USE

Easy Breath Nasal Shower

**WARNING:** Before using the nasal shower, read and carefully keep these instructions for future reference. The nasal shower produces a jet of micronized solution useful in treating upper respiratory tract issues. Ideal for nasal cavity hygiene. It removes secretions, mucus, and phlegm. The particles produced have a specific diameter for the upper airways. The design has been specifically studied to make therapy more practical and

effective by minimizing drug waste.

This product is not a toy. This product must be assembled, maintained, and handled by an adult. Chicco Easy Breath 00012247000000 is a component of the medical device GCE858.

### PREPARATION OF THE NASAL SHOWER:

1. Open the nasal shower by rotating the top counter-clockwise.
2. Remove the gray plastic part and insert the isotonic solution into the transparent tank of the nasal shower without exceeding the maximum level indicated by the marks.
3. Reposition the gray plastic part and close the nasal shower by rotating the two parts clockwise, ensuring they are well sealed.
4. Connect the nasal shower through the air tube to the piston nebuliser.
5. Position the nasal shower at the nostril without inserting it and breathe normally. Close the nostril that is not being treated by pressing with a finger, alternating nostrils during nasal lavage.
6. Activate the nasal shower by turning on the connected nebuliser and place a finger to close the hole to start nasal washing.

### WARNING:

Use only with Chicco piston nebuliser devices. For the correct use of nebuliser devices, refer to the instructions contained in their packaging. **For proper and safe use, the nasal shower should never be inserted into the nostril.** Never insert anything other than common saline physiological solution into the tank or use the nasal shower with substances or liquids different from common saline physiological solution as the device could be irreparably damaged. Do not use this product for purposes other than those for which it was designed as it could be dangerous. The nasal shower should not be used if it is damaged or deteriorated in any part. In case of use by children or individuals with reduced physical or cognitive abilities, they must be supervised and adequately instructed regarding the correct use of the device by a person responsible for their safety.

### CLEANING

Before each use:

- The nasal shower must be emptied and cleaned immediately after each use to avoid deposits that could compromise the proper functioning of the accessory.
- Disassemble the shower into its removable parts. Quickly rinse with running water for at least 10 seconds. Mix a small amount of dish soap and warm water in a clean container. Immerse the nasal shower in the water mixture for about five minutes. Then, scrub the surface with a clean brush. Subsequently, rinse thoroughly with running water for at least 30 seconds to remove detergent residues. For disinfection, immerse in a 2% sodium hypochlorite (NaOCl) disinfectant solution for 15 minutes. Then rinse with clean sterile water twice to completely remove any possible residue from the disinfectant solution. Never use benzene, thinners, or other flammable chemicals for cleaning or disinfection.
- Ensure all components are completely dry before storing them in a clean, dry place, protected from light, away from heat sources.

**WARNING:** The nasal shower and its components cannot be boiled as they would be irreparably damaged.

### ENVIRONMENTAL CONDITIONS FOR USE:

Humidity: 10% to 95% RH  
Temperature: 10°C to 35°C  
Atmospheric pressure from 700hPa to 1060hPa

### STORAGE AND TRANSPORT CONDITIONS:

Humidity: 10% to 95% RH  
Temperature: -25°C to 70°C  
Atmospheric pressure from 700hPa to 1060hPa

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GLOBALCARE MEDICAL TECHNOLOGY CO., LTD.
Arsina S.p.A. Via Saldarini Catelli, 1 - 22070 Grandate - Como - Italy