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EASY BREATH KIT

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EN INSTRUCTIONS FOR USE ⓘ

EASY BREATH KIT

⚠ WARNING: Before using the kit, read and carefully keep these instructions for future reference. A complete kit of all components that allows you to perform therapy in the most suitable way. The masks, designed for adults and children, are made of soft silicone to ensure a better fit to the face and offer maximum comfort during therapy. The Breath-Enhanced ampoule, with high efficiency and double valve system, ensures greater effec-

tiveness of the treatment and less drug waste. This product is not a toy. This product must be assembled, maintained, and handled by an adult. Chicco Easy Breath [REF](#) 00012248000000 is a component of the medical device GCE858.

INCLUDED COMPONENTS:

- 1. Paediatric mask
- 2. Adult mask
- 3. "Breath-Enhanced" ampoule
- 4. Air tube
- 5. Mouthpiece
- 6. Nasal fork
- 7. Replacement filters

Ampoule PREPARATION:

1. Open the ampoule by rotating the upper part counterclockwise (Figure B).
2. Ensure that the drug's conduction cone is correctly positioned on the air conduction cone inside the nebulizer (Figure C).
3. Introduce the medication and/or saline solution into the ampoule, adhering to the doses recommended by the doctor and without exceeding the maximum level that can be contained in the ampoule (Figure D).
4. Close the ampoule by rotating the two parts clockwise, ensuring they are well sealed (Figure E).
5. Connect one end of the air tube to the nebulizer (Figure F) and the other end to the air outlet (Figure G).
6. Connect the recommended component for therapy (mask, mouthpiece...) (Figure H).

CLEANING OF THE AMPOULE AND COMPONENTS:

Quickly rinse all components with running water for at least 10 seconds. Mix a small amount of dish soap and warm water in a clean container. Immerse the disassembled components of the ampoule, the mask, the mouthpiece, and the nasal fork in the water mixture for about five minutes. Then, scrub the surface of the components with a clean brush at least eight times. Subsequently, thoroughly rinse all components with running water for at least 30 seconds to remove residues of liquid dish soap.

For disinfection of masks, immerse the mask in a 2% sodium hypochlorite (NaOCl) disinfectant solution for 15 minutes. Then, immerse the mask in sterile water for 3 minutes and then rinse all mask surfaces with another sterile clean water twice, to completely remove any possible residue from the disinfectant solution. Never use benzene, thinners, or other flammable chemicals for cleaning or disinfection.

ENVIRONMENTAL CONDITIONS FOR USE:

Humidity: 10% to 95% RH
Temperature: 10°C to 35°C
Atmospheric pressure from 700hPa to 1060hPa

STORAGE AND TRANSPORT CONDITIONS:

Humidity: 10% to 95% RH
Temperature: -25°C to 70°C
Atmospheric pressure from 700hPa to 1060hPa

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