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## Easy Breath NASAL IRRIGATOR

**WARNING:** Read carefully and keep these instructions for future reference.

### 1. Intended use

The device is used for nasal irrigation.

### 2. Product description

It consists of an olive-shaped tip and a syringe. The syringe itself

comprises a graduated container cylinder and a piston with a rubber grommet.

### 3. Instructions for use

• Open the individual packaging and remove the device. • Draw saline into the syringe, reading the correct scale for the required dose. • Holding the syringe in the hand, place the flushing connector into the nasal cavity. • Tilt the head slightly forward, open the mouth and push the mandrel to flush in the saline. • Discard the used equipment into the waste container after flushing.

### 4. Contraindication

• Avoiding nasal irrigation as much as possible during acute attacks of allergic rhinitis. • Avoiding nasal irrigation as much as possible in the presence of active bleeding in the nasal cavity.

### 5. Side effects/Potential complications

No side effects/potential complications were identified with the product.

### 6. Precautions and warnings

Please pay attention to the following instructions during use, as it is a disposable medical device: 1. The user should be positioned so that the saline enters the nasal cavity and then flows out the other side or the mouth. 2. When flushing, care should be taken to push the plunger with moderate and even pressure and try to avoid using too much force to prevent damage to the mucous membrane of the nasal cavity. 3. Patients should try to breathe through their mouths as much as possible during the application process and open them as wide as possible without swallowing saline to prevent choking. 4. For pediatric use, this product is designed to be administered by an adult caregiver, ensuring the safety and well-being of the child. 5. Users need to be careful to avoid using the product too frequently for flushing. 6. Dispose of the product after use according to local regulations. 7. Only use the supplied 5ml syringe. 8. do not exceed with the quantity of saline. 9. check that the detachable parts are fitted correctly. 10. Only use in the presence of an adult and keep out of reach of children. The product contains small parts that can be swallowed. Warning! Suffocation risks. 11. Report any serious accident to the manufacturer and to the competent authorities for the given country.

### 7. Storage, transportation and maintain

Do not use after the expiry date. Susceptible to light. The product must be stored in its original packaging, in a fresh and dry place and in the shelter of light.

 Keep away from sunlight	 Fragile, handle with care
 Not produced with/Does not contain natural latex rubber	 Do not use if package is damaged and consult instructions for use
 Recyclable	 Single use. Do not re-use
 Take care of the environment	 Medical device
 Model number	 Catalog number
 Use-by date	 Manufacturer
 Date of manufacture	 Batch code
 Consult instructions for use	 Temperature limit
 Caution	 Humidity limit
 Symbol of compliance with REGULATION (EU) 2017/745	 Keep dry

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Artsana S.p.A.
Shanghai International Holding Corp.GmbH ( Europe )
Shanghai Kindly Enterprise Development Group Co., Ltd.
Artsana S.p.A. Via Saldarini Catelli, 1 - 22070 Grandate - Como - Italy - www.chicco.com