IT Limbazione salina isotonica

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Read instructions before use.

Shelf life: 5 years INDICAZIONI

Isotonic saline solution is used for nasal lavage, for aerosol therapy and for external eye washes. When used for nasal lavage, it cleanses and moisturizes the nasal mucosa helping drain secretions and eliminate excess mucus, in the event of infections of the upper airways. Helps prevent otitis and sinusitis. Creates the conditions for easy and correct breathing, which in younger children facilitates a good night's sleep and better feeding. When used in aerosol therapy it moisturizes the airways helping to expel sputum, in the event of infections of the lower airways. It is also used for external eye cleansing, in order to prevent bacterial or viral infections. It has no intrinsic contraindications to its use if you follow the instructions for use correctly. The single-dose vials are hygienic and practical.

INSTRUCTIONS

Always wash your hands before opening the bottle.

Nasal hygiene

1 - Twist the tab to open the sterile vial (fig. 1). 2 - Place the child on its side using your hand to support the head (fig. 2). Gently rest the tip of the vial against the nostril and delicately press the solution out of the vial. 3 - Wait a moment for the solution to act. Lift the child and clear any secretions with a tissue or nasal aspirator (fig. 3). Repeat the operation with the other nostril. 4 - Dispose of the vial after use.

External cleansing of the eyes

Place a few drops of saline solution on a sterile piece of gauze (one for each eye) and gently clean the outside of each eyelid, avoiding direct contact with the eyes.

For aerosol therapy

Twist the tab to open the sterile vial (fig. 1). PhysioClean can be used with any aerosol therapy device. Comply with the instruction manual for the given device.

COMPOSITION

Sodium Chloride 0,9 g, distilled water as required for 100 ml. STERILE.

WARNINGS

- Use as instructed by a doctor/paediatrician.
- For nasal use, do not ingest. Not injectable.
- Store in a dry, cool place. Do not expose to light and heat.
- Store the instructions for as long as the product is used.
- The product is in single-dose vials. It is recommended to use it only once as sterility is no longer guaranteed.
- Do not use the vial after its expiration date.
- Dispose of the vials properly.
- Do not use the same single-dose vial for nasal and eye hygiene.
- For children less than 2 months of age, or patients with significant respiratory illness, or in the event of frequent nose bleeds, or neurological or musculoskeletal diseases or maxillofacial trauma consult your physician before use.
- Keep out of reach of children.
- Report any serious accident occurring in connection with the device to your competent authority and to the manufacturer.
- Do not use if the vial is damaged.

MD	Medical Device
CE	CE marking
REF	Product catalogue reference code
REF	Legal Manufacturer
LOT	Production Batch
REF	Expiring date
REF	Instruction manual
REF	Do not use if package is damaged
REF	Sterile
REF	Do not re-use
REF	Do not re-sterilise
REF	Unique identifier of the equipment
REF	Distributor

Date of manufacture: first 4 digits of LOT (e.g. 23039999; 23=2023; 03=March)

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PIKDARE S.p.A.
Artansa S.p.A.